



5085 Corporate Exchange Blvd. S.E.
Grand Rapids, MI 49512 • 616.656.2900

TABLE RELAY BOX DIP SWITCH SETTINGS INSTRUCTIONS FOR USE

The Skytron Relay box offers a wide variety of configurations. The dip switch settings for each model offered in the exchange program are listed below and on the back page.

WARNING

Avoid exposure to electrical hazards and unexpected energizing or startup of the equipment when replacing the relay box.

To avoid electrical hazards, unplug the equipment from the energy source and ensure the plug is under the exclusive control of the individual performing the maintenance or repair.









Anti-Static guarding is required in replacement of the relay box.

IMPORTANT! All table functions must be tested after the relay box is installed.

Please contact Skytron technical support at:
techinquiries@skytron.us
for questions regarding dip switch settings.

RB-00 Series Relay Box Dip Switch Settings



3501		
	DIP1	DIP2
3600		
	DIP1	DIP2
6700		
	DIP1	DIP2
6701		
	DIP1	DIP2