

10 INCH FOOT EXTENSION WITH CLAMPS - INSTRUCTIONS FOR USE

The 10 Inch Foot Extension with Clamps is an accessory for a SKYTRON surgical table that is used to support the feet and legs of the patient during a procedure. This accessory can be added to the table to extend the length after the leg section. The accessory can also be installed perpendicular to the table. Use the accessory as a foot board in the perpendicular position.

Length	17 in
	432 mm
Width	22 in
	559 mm
Weight	7.4 lbs
	3.36kg

Adjacent Installation

Use the instructions below to install the foot extension on the end of the leg section.

1. Install two EZ grip bar clamps on the side rails on the surgical table.



Install one clamp on each side of the surgical table.

- (a) Push the bottom of the gravity stop up on the side rail of the surgical table.



Figure 1. Gravity Stop

- (b) Slide the clamp over the gravity stop, onto the side rail on the surgical table.
- (c) Release the gravity stop.

OR

- (a) Locate the rail cut outs in the side rail on the surgical table.
- (b) Place the clamp over the side rail on the surgical table at a 45 degree angle over the rail cut outs.



Figure 2. Clamp Over Rail Cut Outs

- (c) Lower the clamp so the clamp is parallel to the rail.
- (d) Slide the clamp into place.

2. Insert the foot extension horizontally into the clamp.
3. Tighten the clamp with the handle on each side.
4. Install the desired pad on the foot extension.
5. Verify the extension is secure. If not, Tighten the clamp further.

6. Complete the sterile drape procedure on the foot extension, if required.



WARNING

Be aware that the installation of the 10 inch foot extension can cause instability if a patient weight load is uneven. To prevent serious injury, make sure the weight distribution of the patient is even and stable.

Perpendicular Installation

Use the instructions below to install the foot extension perpendicular to the leg section.

1. Install two EZ grip bar clamps on the side rails on the surgical table.



Install one clamp on each side of the surgical table.

- (a) Push the bottom of the gravity stop up on the side rail of the surgical table.



Figure 3. Gravity Stop

- (b) Slide the clamp over the gravity stop, onto the side rail on the surgical table.
- (c) Release the gravity stop.

OR

- (a) Locate the rail cut outs in the side rail on the surgical table.
- (b) Place the clamp over the side rail on the surgical table at a 45 degree angle over the rail cut outs.



Figure 4. Clamp Over Rail Cut Outs

- (c) Lower the clamp so the clamp is parallel to the rail.
 - (d) Slide the clamp into place.
2. Insert the foot extension into the top of the clamp.



Figure 5. Clamp with Perpendicular Installation

3. Tighten the clamp with the handle on each side.
4. Verify the extension is secure. If not, Tighten the clamp further.
5. Complete the sterile drape procedure on the foot extension, if required.

**WARNING**

Be aware that the installation of the 10 inch foot extension can cause instability if a patient weight load is uneven. To prevent serious injury, make sure the weight distribution of the patient is even and stable.

Perpendicular Installation

To remove the foot extension from a surgical table, follow the instructions below:

1. Loosen the clamp on each side of the foot extension. Use the handle to loosen the clamp.
2. Remove the foot extension from the foot section of the surgical table.
3. Remove both clamps from the side rails on the surgical table.

4. Store the clamps with the foot extension.

**WARNING**

Be aware that the installation of the 10 inch foot extension can cause instability if a patient weight load is uneven. To prevent serious injury, make sure the weight distribution of the patient is even and stable.

Cleaning Recommendation



Always follow the current Association of periOperative Registered Nurses (AORN) Journal Guidelines to make sure the proper cleaning and disinfection procedure is followed.

Follow the procedures below to clean the 10 inch foot extension between cases.

**WARNING**

Always follow the standards for blood-borne pathogens from the Occupational Safety and Health Administration (OSHA). Use the recommended protective clothing, gloves, masks and eye protection to clean the table extension.

1. Remove major contaminants from the accessory with disposable materials. Follow the appropriate biohazard waste disposal procedures.
2. Remove the accessory and place it on a flat surface for cleaning.

CAUTION

Thoroughly read and follow the manufacturer's directions for all cleaning fluids. DO NOT use cleaners containing phenolics.

3. Use a ready-to-use detergent diluted as required by the manufacturer. Make sure the active ingredients of the product are compatible with the materials used in the 10 inch foot extension.
4. Apply cleaning fluid liberally to the top and sides of each 10 inch foot extension.
5. Wipe with a clean, lint-free cloth.
6. Wipe until all the moisture and cleaning fluid is removed from the accessory.
7. Apply cleaning fluid liberally to the bottom of the each 10 inch foot extension.
8. Wipe with a clean, lint-free cloth.
9. Wipe until all the moisture and cleaning fluid is removed from the bottom of the accessory.
10. Allow the accessory to dry.

Disinfection

The following antiseptic solutions are approved for use on the 10 inch foot extension:

- Sodium Hypochlorite 6% diluted to 0.1% (halogen containing compound)
- HypoAlcohol (iodine color removing agent)
- Chlorhexidine (chlorhexidine gluconate 0.5%)
- Benzalkonium Chloride (invert soap 10%)
- Povidone Iodine
- Ethanol 80%
- Oxydol (hydrogen peroxide)
- Isopropyl Alcohol (IPA) 99.5%

Use the following steps to disinfect the 10 inch foot extension.


Remove all pads from the 10 inch foot extension, if applicable.

Apply disinfectant to a clean, lint-free cloth.

Wipe the top, sides and bottom of the accessory with the cloth.



Make sure there is enough disinfectant on the cloth so the cloth does not dry out before the entire accessory is wiped and disinfected.

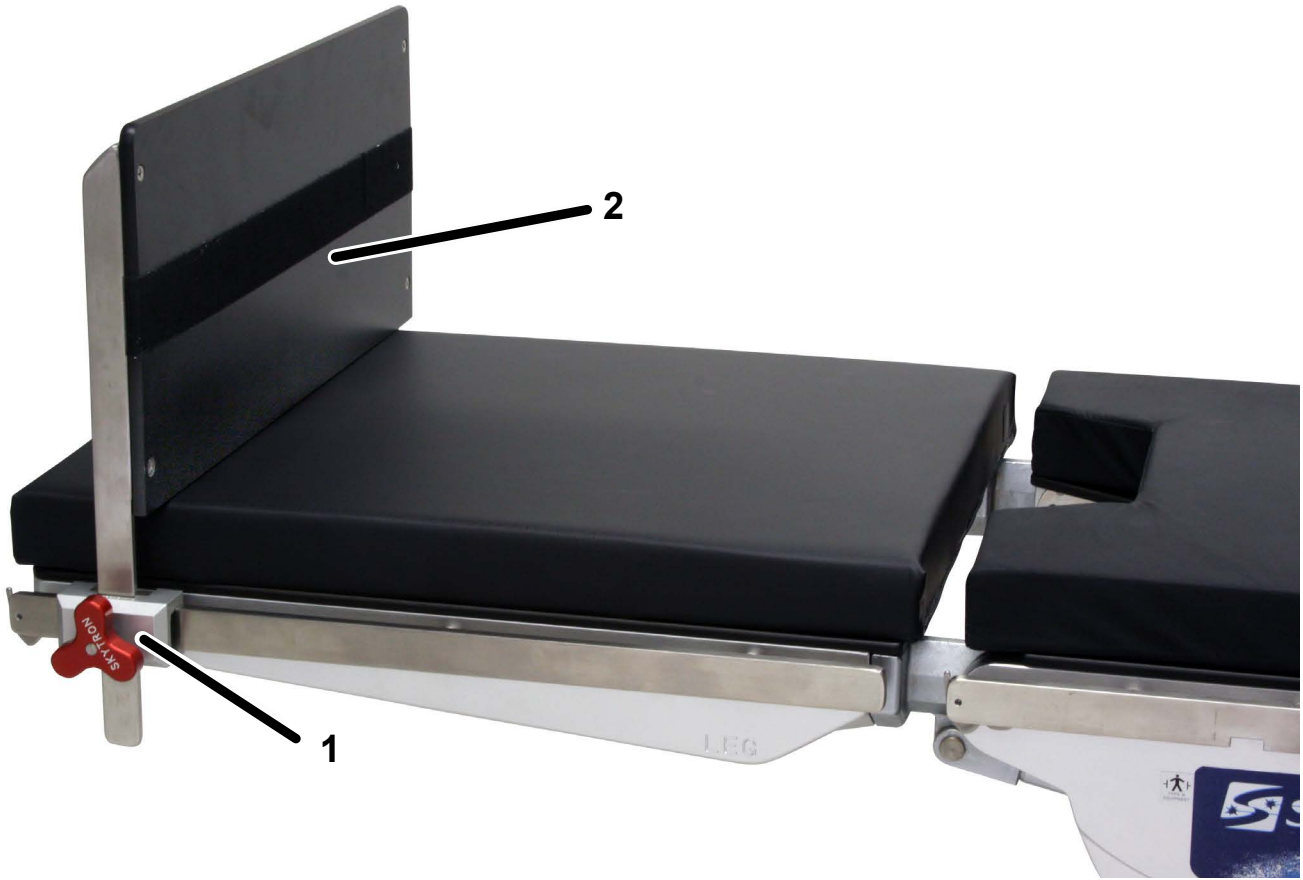

Item	Quantity	Part Number	Description
1	2	6-030-05-1	EZ GRIP BAR CLAMP
2	1	4-030-13-T	TOP
PADS:			
	NS	4-030-13-2P	2" PAD
	NS	4-030-13-2S	2" SOF-PAD
	NS	4-030-13-4P	4" PAD
	NS	4-030-13-4S	4" SOF-PAD

NOTES

NOTES

REVISION HISTORY

Date	Revision	Revision History
03/28/2016	0	Initial Release

Distributed by:
SKYTRON
5085 Corporate Exchange Blvd. S.E.
Grand Rapids, MI 49512
616-656-2900
www.skytron.us

Manufactured by:
Freedom Manufacturing Inc
5320 6 Mile C. NW
Comstock Park MI 49321
616-647-9200
www.freedom-mfg.com

