

REPLACEMENT PENDANT CONTROLS - INSTRUCTIONS FOR USE

Replacement pendant controls offered by Skytron are a cost-saving solution without any sacrifice in quality. Built with the same quality construction that is standard in Skytron products, the replacement pendant program will provide cost savings and reliable equipment from the original manufacturer.

Replacement Pendant Controls

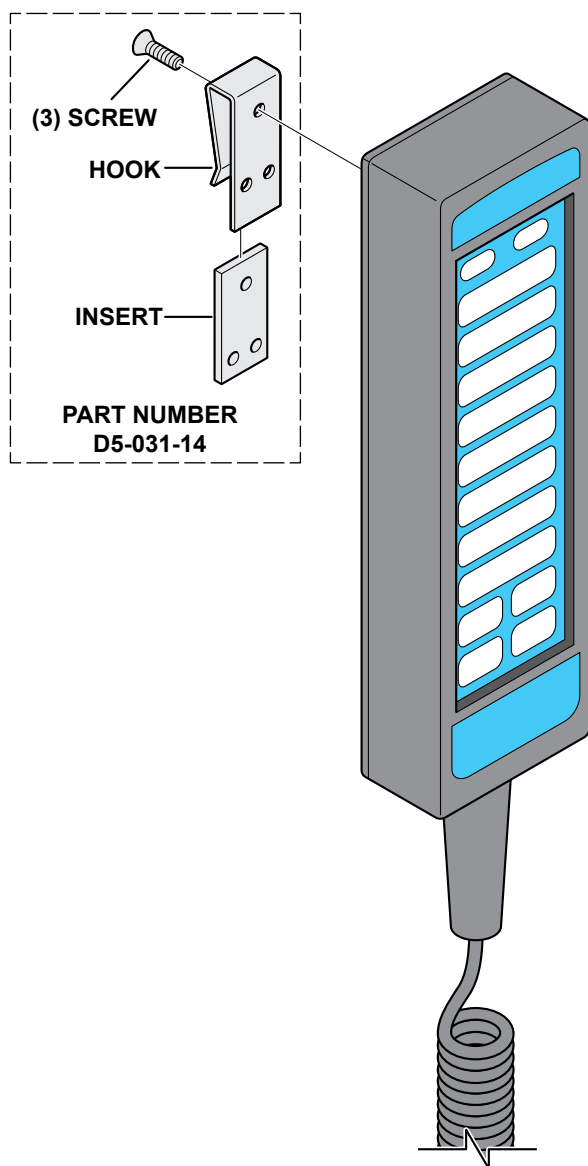


Table Model	Pendant Control Part Number	Cord Assembly Part Number
3000	U-D1-065-01-2	D5-034-18-1
3001	U-D5-065-01-2	D5-035-12
3002	U-D5-066-01-2	D5-066-03
3003	U-D5-066-01-3	D5-066-03
3100A	U-D5-060-30-2	NA
3100N	U-D5-060-21-2	NA
3500	U-D5-035-02-2	NA
3500B	U-D5-035-17-2	D5-034-18-1
3501B	U-D5-035-17-2	D5-034-18-1
3501B	U-D5-035-18-2	D5-034-18-1
3503	U-D5-036-01-3-1	M001962A
3600B	U-D5-036-01-2	D5-034-18
3603	U-D5-036-01-3	M001962A
6001	U-D5-031-63-2	NA
6001NB	U-D5-031-65-2	NA
6002	U-D5-031-74-2	NA
6300 Series	U-D5-063-01-2	D5-034-18
6500	U-D5-032-29-2	NA
6500NB	U-D5-032-46-2	NA
6600	U-D5-034-01-2	D5-034-18
6600B	U-D5-034-03-2	D5-034-18
6700B /6701	U-D5-067-16-2	D5-034-18

NA = Not available separately.

Cleaning Pendant Controls

CAUTION

DO NOT submerge the pendant control in any water or cleaning solutions!

1. Apply cleaning solution to the pendant control and cord.
2. Using a clean, damp, lint-free cloth, wipe the pendant control to remove the cleaning solution.
3. Using a clean, dry, lint-free cloth, wipe the pendant control and cord to remove all moisture.
4. Clip the pendant control to the side rail for storage.



Disinfecting Pendant Controls

Acceptable Solutions:

- Sodium Hypochlorite 6% diluted to 0.1% (halogen containing compound)
- Hypo Alcohol (iodine color removing agent)
- Chlorhexidine (chlorhexidine gluconate 0.5%)
- Benzalkonium Chloride (invert soap 10%)
- Povidone Iodine
- Ethanol 80%
- Oxydol (hydrogen peroxide)
- Isopropyl Alcohol (IPA) 99.5%

NOTICE

Ensure disinfectant does not dry when wiping the Pendant Control.

1. Apply a proper amount of disinfectant on a clean, lint-free cloth. Thoroughly wipe the top, sides, and bottom of the pendant control with the cloth.
2. Wipe the pendant control with a dry, clean, lint-free cloth within 15 minutes after disinfecting or as directed for the disinfecting agent.

Revision History

Date	Revision	Revision History
10/11/2019	0	Initial Release

Distributed by:

US - SKYTRON

5085 Corporate Exchange Blvd. S.E.

Grand Rapids, MI 49512 (616) 656-2900

www.skytron.us

Manufactured by:

Mizuho Corporation

3-30-13 Hongo, Bunkyo-ku

Tokyo 113-0033

JAPAN



The base language for this document is ENGLISH. Any translations must be from the base language document.

Printed copies are not controlled documents.

Although current at the time of publication, SKYTRON'S policy of continuous development makes this document subject to change without notice. If a current document is required, contact your local SKYTRON representative or contact SKYTRON directly at the distribution addresses listed above.



5085 Corporate Exchange Blvd. S.E.
Grand Rapids, MI 49512 • 616.656.2900