



PN 2-010-06

## 12" CARBON FIBER ARM SURGERY BOARD - INSTRUCTIONS FOR USE



### Introduction

The Skytron arm surgery board was designed for use when unrestricted imaging through the length and width of the board is needed. This 12" (305mm) wide carbon fiber arm surgery board uses a special clamping system that is compatible with Skytron 40" (1016mm) carbon fiber extensions or Skytron 3000 series imaging tables. The pad is available separately.

### Accessory Dimensions

Dimension	English	Metric
Width	12"	305mm
Width at Clamps	16"	406mm
Length	31"	787mm
Weight	7.6 lbs	3.44 kg

### Installation

a. Ensure that both clamps are open far enough to allow the 12" carbon fiber arm surgery board to be placed over the edge of the 40" extension or 3000 series imaging table top.

- The clamp handles should be facing the floor.
- The convex ridge on the clamp should be aligned to the concave groove on the underside of the extension or table top.

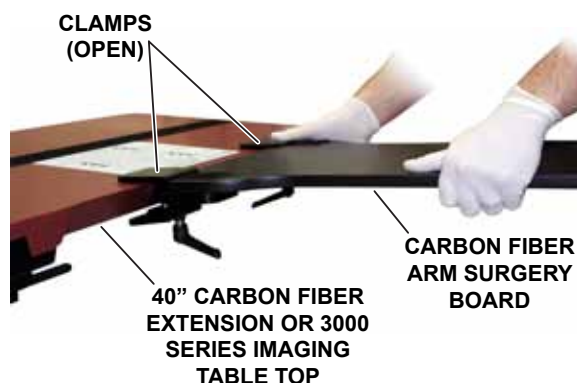


Figure 1. Installation

b. Tighten both clamp handles to secure the arm surgery board in place, then pull on the board to ensure that it is satisfactorily attached (Figure 2).



Figure 2. Secure Arm Surgery Board

c. If the armboard feels loose, recheck the tightness of each clamp handle.

- d. Install the desired pad (velcro side down) on the arm surgery board.

### Before Use

The arm surgery table must be sterile draped prior to any procedural use.

### Removal

- a. Remove the pad from the arm surgery board.
- b. Loosen the two (2) clamp handles.
- c. Slide the arm surgery board off the 40" extension or 3000 series imaging table top.

### Cleaning and Disinfecting

#### NOTICE

Always follow current AORN Journal Guidelines to ensure proper cleaning and disinfection procedures.

The following procedures should be followed when cleaning and disinfecting the surgical table accessory between cases:

- a. Place the table top in a level position.



#### WARNING

**Always follow OSHA blood-borne pathogens standards for protective clothing, including gloves, masks, and eye protection when cleaning this accessory.**

- b. Remove major contaminants with disposable materials following appropriate biohazard waste disposal procedures.
- c. Remove pads and place on a flat surface for cleaning.



#### WARNING

**Pads should be treated as hazardous waste and disposed of properly.**

- d. Examine pads for signs of wear or damage, indicating a compromise to the pad's seal. Replace any worn or damaged pads as required.

### CAUTION

Thoroughly read and follow the manufacturer's directions for all cleaning fluids. **DO NOT** use cleaners containing phenolics.

- e. Wash table pads and the arm surgery board accessory with a generous application of neutral soap suds and lukewarm water. Rinse with water and dry. **DO NOT** immerse.

### CAUTION

Before replacing pads on the arm surgery board, make sure the pads and all mating surfaces are completely dry. Moisture trapped between the pads and mating surfaces may cause distortion of the arm surgery board.

- f. When disinfecting:
  - Use Lysol I.C. disinfectant spray (PN 95029-000) by National Laboratories, Montvale, NJ per manufacturer's instructions.

or

- Use a diluted bleach solution of 10-parts water to 1-part bleach. Wipe dry with a clean cloth. **DO NOT** immerse. Bleach **MUST** be diluted to 10:1 ratio.

- g. When cleaning and disinfecting is complete, replace all pads.

### Service

Maintenance can be performed by trained maintenance personnel using SKYTRON authorized replacement parts and service techniques. Service instructions and parts are available from SKYTRON.

To obtain service instructions, replacement parts, or factory service, contact your local SKYTRON representative.

Replacement Parts



Item	Part Number	Description	Quantity
1	6-070-07-4	LEVER	2
2	2-010-05-2S	2" Pad	1
	2-010-05-3S	3" Pad	1
	2-010-05-4S	4" Sof-Pad	1


**Authorized Distributors:**

**U.S. - SKYTRON**

5085 Corporate Exchange Blvd. S.E.  
Grand Rapids, MI 49512

**EUROPE - SKYTRON EUROPE BV**

FLORESSTRAAT 52  
8022 AD ZWOLLE  
THE NETHERLANDS

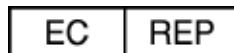

**Manufactured by:**

SchureMed  
50 Messina Drive  
Braintree, MA 02184

