



5085 Corporate Exchange Blvd. S.E.
Grand Rapids, MI 49512 • 616.656.2900

3.5 INCH TABLE SIDE EXTENSION - INSTRUCTIONS FOR USE

PN 3-060-04

Introduction

The 3.5 Inch Table Side Extension, shown in Figure 1, clips to the side of the table using an integrated clamp system to lock on to the side rail. The 3.5 Inch Table Side Extension extends the table surface to accommodate larger patients, and has an additional side rail to attach other accessories.



Figure 1. 3.5 Inch Table Side Extension

Typical Dimensions

Width: 3.5 inches (89 mm)

Length: 15 inches (381 mm)

Weight: 5.5 lbs (2.5 kg)

Installation

Using the "U" shaped bar on the under side of the extension, open the clamps. (See Figure 2.)



Figure 2. Open Clamps

At the same time attach the table side extension to the table side rail at a slight angle. (See Figure 3.) Release the lever once in place, and check to ensure the extension is secured by lightly pulling from multiple angles to ensure proper attachment.

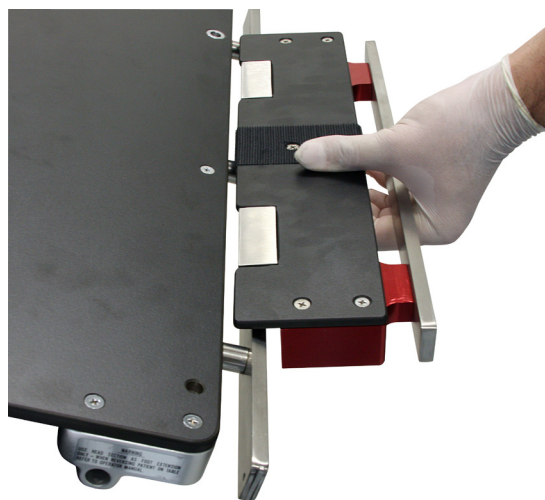


Figure 3. Attach Table Side Extension

Cleaning / Disinfecting Recommendations

NOTICE

Always follow current AORN Journal Guidelines to ensure proper cleaning and disinfection procedure.

Use the table cleaning and disinfection procedures to clean and disinfect this surgical table accessory between cases.

After cleaning, store in accordance with standard hospital operating procedures to maintain cleanliness. Inspect before and after each use.

Replacement Parts



Item	Part Number	Description	Quantity
1	3-060-04-1	Top, Table Side Extension	1
2	3-060-01-2P	2" Pad	1
	3-060-01-2S	2" Sof-Pad	1
	3-060-01-3S	3" Sof-Pad	1
	3-060-01-4P	4" Pad	1
	3-060-01-4S	4" Sof-Pad	1
	3-060-01-5S	5" Sof-Pad	1

Distributed by:

SKYTRON
5085 Corporate Exchange Blvd. S.E.
Grand Rapids, MI 49512
616-656-2900
www.skytron.us



Manufactured by:

Freedom Manufacturing
5320 6 Mile Ct. N.W.
Comstock Park, MI 49321