



5085 Corporate Exchange Blvd. S.E.
Grand Rapids, MI 49512 • 616.656.2900

PN 4-030-40

40 INCH CARBON FIBER TABLE EXTENSION - INSTRUCTIONS FOR USE

The 40 Inch Carbon Fiber Table Extension is a table accessory designed to improve imaging capabilities with a lightweight material. This accessory works with various Skytron tables and pads. The Skytron tables that accommodate this table extension are:

- Model 3500 Series
- Model 3600 Series
- Model 6300 Series
- Model 6600 Series
- Model 6700 Series

Length	40 in
	1016 mm
Width	19-5/8 in
	498 mm
Height	3-7/8 in
	98 mm
Weight	16.65 lbs
	7.55 kg

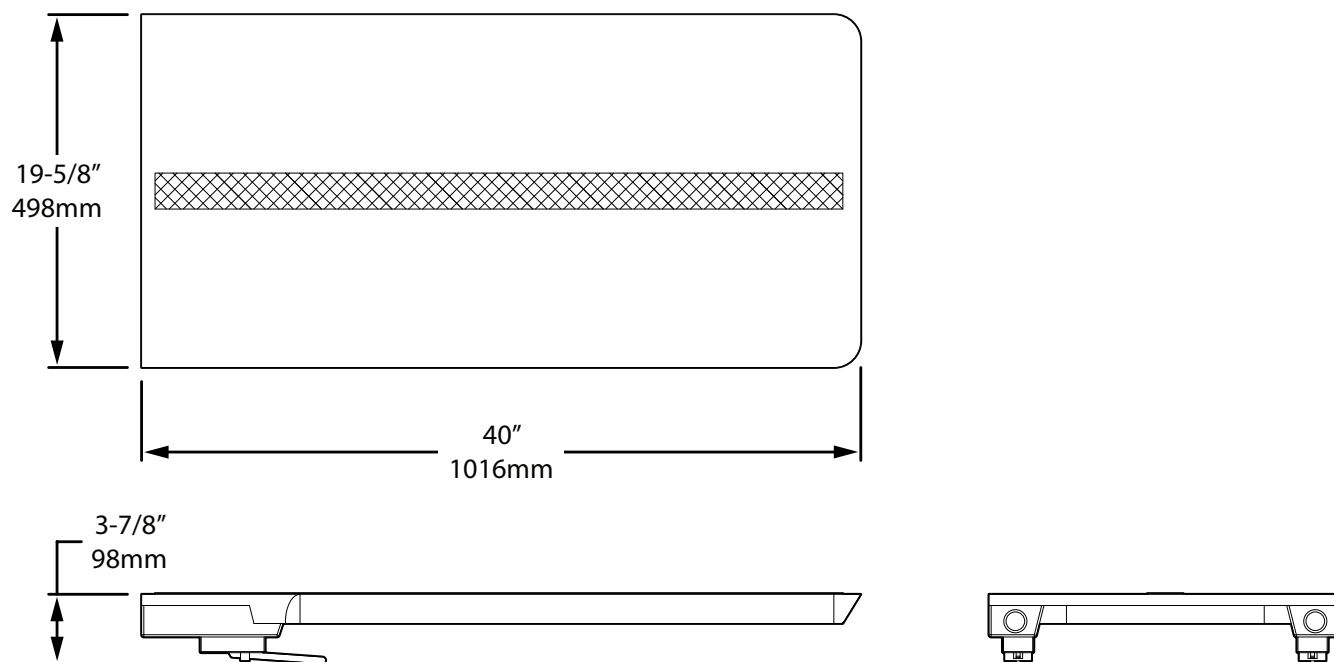


Figure 1. Specifications for the 40 Inch Carbon Fiber Table Extension

Install the Table Extension on the Leg Section

1. Remove the pad set.
2. Remove the x-ray top from the table.
3. Adjust the height of the table top to a safe and comfortable height required to remove the leg section.
4. Simultaneously squeeze the two release levers on the bottom of the leg section and pull the leg section off the attaching pins.



WARNING

The leg section weighs approximately 20 pounds (9.07 kilograms). Safely handle the leg section during removal to prevent injury.

5. Install the table extension on the attaching pins.
6. Push the table extension until it rests against the pin collar and clicks into place.
7. Pull on the table extension to verify it is secure.



WARNING

Do NOT use this table extension if it does not properly install on the attaching pins.

8. Install the appropriate pad.



The pads are not included with the purchase of the 40 Inch Carbon Fiber Table Extension.

Install the Table Extension on the Back Section

1. Remove the pad set.
2. Remove the x-ray top from the table.
3. Adjust the height of the table top to a safe and comfortable height required to remove the head section and the back section.
4. Loosen the locking knobs on the bottom of the head section.
5. Remove the head section.
6. Simultaneously squeeze the two release levers on the bottom of the back section and pull the leg section off the attaching pins.



WARNING

The table sections are heavy. Safely handle the table sections during removal to prevent injury.

7. Install the table extension on the attaching pins.

8. Push the table extension until it rests against the pin collar and clicks into place.
9. Pull on the table extension to verify it is secure.



WARNING

Do NOT use this table extension if it does not properly install on the attaching pins.

10. Install the appropriate pad.



The pads are not included with the purchase of the 40 Inch Carbon Fiber Table Extension.

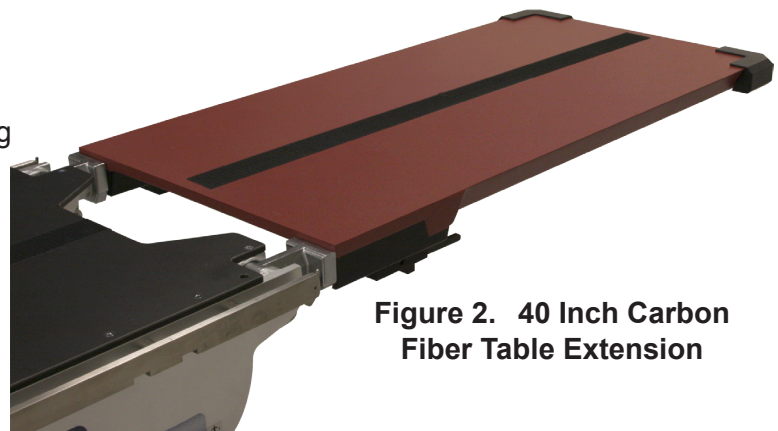


Figure 2. 40 Inch Carbon Fiber Table Extension

Remove the Table Extension

1. Adjust the height of the table top to a safe and comfortable height required to remove the table extension.
2. Remove the pad set.
3. Simultaneously squeeze the two release levers on the bottom of the table extension and pull it off the attaching pins.

Important User Information

The operator of this accessory is responsible to prevent damage to the table and surrounding equipment. The operator is also responsible to prevent injury to the patient and the staff.

All personnel must review the following precautions before operating the table.



CAUTION



Check for obstacles before the surgical table is lowered or tilted.



WARNING

Do NOT use this table extension if it does not properly install on the attaching pins.



WARNING

Do NOT use this table extension if any parts are missing, broken or have excessive wear. Contact your Skytron representative to request service related to this product.



WARNING

Do NOT use this table extension if any cracks or chips have exposed any carbon fiber elements. Exposure to the carbon fiber elements may cause skin irritation. Contact your Skytron representative to request service related to this product.

Weight Limits per Table Model




40 Inch Carbon Fiber Extension

Patient Weight Limitations

Caution: Table brakes must remain locked to the floor.

Caution: Not for use on Skytron 6500 Series Tables.

Model 3500(B)

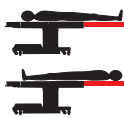

Used in place of leg section limited to a 500 lb. patient



Used in place of leg section w/patient reversed limited to a 300 lb. patient

Model 6300 Series




Used in place of leg section limited to a 600 lb. patient



Used in place of leg section w/patient reversed limited to a 250 lb. patient

Models 3501B, 3501C and 3502

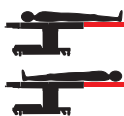

Used in place of leg section limited to a 600 lb. patient



Used in place of leg section w/patient reversed limited to a 300 lb. patient

Model 6600 Series



Used in place of back section limited to a 600 lb. patient




Used in place of leg section limited to a 600 lb. patient



Used in place of leg section w/patient reversed limited to a 300 lb. patient

Model 3600 Series

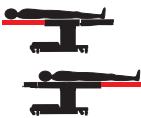

Used in place of back section limited to an 800 lb. patient



Used in place of leg section limited to an 800 lb. patient



Used in place of leg section w/patient reversed limited to a 300 lb. patient

Models 6700 Series



Used in place of back section limited to an 800 lb. patient



Used in place of leg section limited to an 800 lb. patient



Used in place of leg section w/patient reversed limited to a 500 lb. patient

Preventive Maintenance

Complete the preventive maintenance identified below. Regular preventive maintenance makes sure the accessory operates correctly. Preventive maintenance should only be completed by qualified and trained personnel from Skytron.

- Complete a general, visual examination during cleaning. Check for:
 - Loose bolts
 - Loose parts
 - Cracks
 - Chips

Complete the necessary repairs or replace the accessory.

- Complete a periodic inspection of the following:
 - Operation of the release lever
 - Velcro
 - Top surface



WARNING

Do NOT use this table extension if any parts are missing, broken or have excessive wear. Contact your Skytron representative to request service related to this product.



WARNING

Do NOT use this table extension if any cracks or chips have exposed any carbon fiber elements. Exposure to the carbon fiber elements may cause skin irritation. Contact your Skytron representative to request service related to this product.

Cleaning Recommendation



Always follow the current Association of periOperative Registered Nurses (AORN) Journal Guidelines to make sure the proper cleaning and disinfection procedure is followed.

Follow the procedures below to clean the 40 Inch Carbon Fiber Table Extension between cases.

1. Place the table top in a level position.



WARNING

Always follow the standards for blood-borne pathogens from the Occupational Safety and Health Administration (OSHA). Use the recommended protective clothing, gloves, masks and eye protection to clean the table extension.

2. Use disposable cleaning materials to remove major components. **Follow the appropriate procedures to dispose of biohazard waste.**
3. Remove pad sets from the table extension. Clean and disinfect the pad sets between cases.
4. Examine pads for wear or damage. Verify that the seal on the pad is not compromised.



CAUTION

Read and follow the directions from the manufacturer for all cleaning fluids. Do NOT use cleaners containing phenolics.

5. For soils and stains:
 - Wash the Carbon Fiber Table Extension with a large amount of neutral soap suds and lukewarm water. Rinse with water. Dry the table extension. **Do NOT** immerse the table extension in water.

OR

- Wash the Carbon Fiber Table Extension with a diluted bleach solution. The bleach must be 10 parts water to 1 part bleach. Wipe the accessory dry with a clean cloth. **Do NOT** immerse the table extension in water. Bleach **MUST** be diluted to the 10:1 ratio.

6. To disinfect:

- Use Lysol I.C. Disinfectant Spray by National Laboratories, Montvale, NJ per manufacturer's instructions. Reference: National Laboratories Item # 95029-000

OR

- Wash the Carbon Fiber Table Extension with a diluted bleach solution. The bleach must be 10 parts water to 1 part bleach. Wipe the accessory dry with a clean cloth. **Do NOT** immerse the table extension in water. Bleach **MUST** be diluted to the 10:1 ratio.





CAUTION

Make sure the pad and the table extension are both dry before the pad is placed on the table extension again. Moisture trapped between the pad and the table extension can cause distortion of the top section.


7. Replace the pads on the table extension.
8. When pads need to be disposed of, treat them as hazardous waste. Dispose of the pads properly.

Labels

The location of the labels on this accessory is identified in the images below.

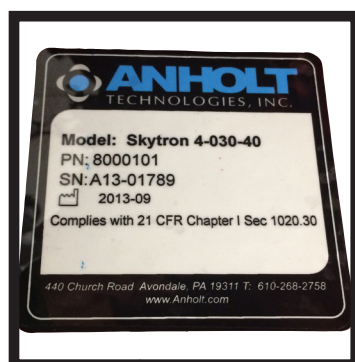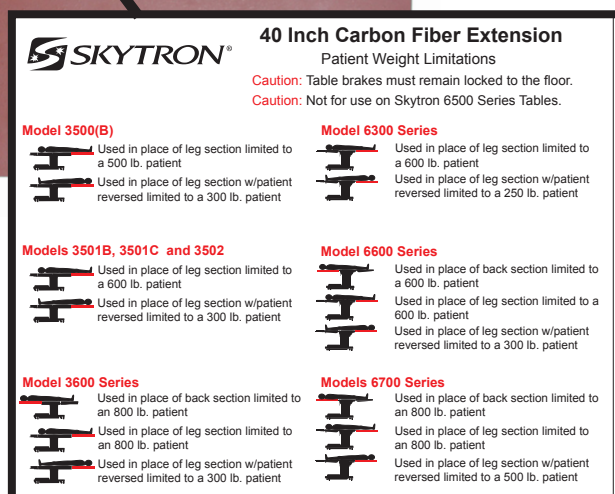

Replacement Parts

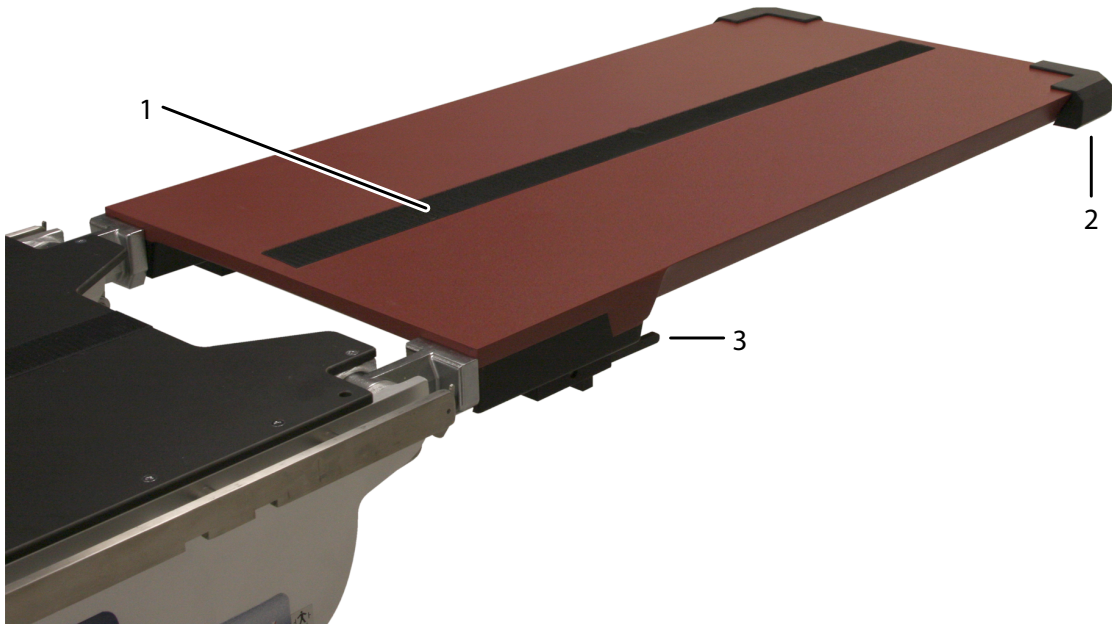

Item	Quantity	Part Number	Description
1	NA	D3-010-18-H	VELCRO/ HOOK, PER FOOT
2	2	4-030-36-B	BUMPER
3	1	4-030-40-2	RELEASE HANDLES, PAIR
NS	1	4-030-40-1	LABEL, WEIGHT LIMITATIONS, 40"

PADS:			
	NS	4-030-40-2S	2" SOF-PAD
	NS	4-030-40-3S	3" SOF-PAD
	NS	4-030-40-4S	4" SOF-PAD

Date	Revision	Revision History
01/25/2016	0	Initial Release

Distributed by:
SKYTRON
 5085 Corporate Exchange Blvd. S.E.
 Grand Rapids, MI 49512
 616-656-2900
www.skytron.us

Manufactured by:
 Anholt Technologies, Inc
 440 Church Rd
 Avondale, Pennsylvania 48430
 610-268-3480
www.anholt.com

