




5085 Corporate Exchange Blvd. S.E.
Grand Rapids, MI 49512 • 616.656.2900

P.N. 3-060-08

5" TABLE SIDE EXTENSION INSTRUCTIONS FOR USE

5" Table Side Extension (with side rails, without pad)

The SKYTRON 5" Table Side Extension is designed to fit on the back, seat, and leg sections of compatible SKYTRON tables. The 5" Table Side Extension will increase the table width up to 30" (762mm). It is recommended that 6 units are used to fully outfit one surgical table. The 5" Table Side extension can be used in the weight management and distribution to support a patient weight of up to 1,200lbs (545kg).

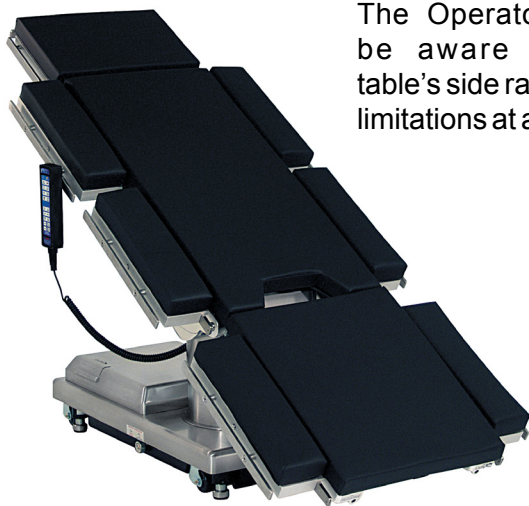

The Operator must be aware of the table's side rail weight limitations at all times.

The 5" Table Side Extension is compatible with the following SKYTRON Tables: 6002, 6500 Series, 6600 Series, 6700 Series, 3500 Series, and 3600B. The 5" Table Side Extension will *not* fit on the SKYTRON 1700 and 6300 tables




CAUTION




Due to the potential for increased loads this accessory can place on the table side rail, it is recommended that a maintenance program be established to check side rails for tightness on a consistent basis.

1. Locate standoffs on table side rail and align with notches on side extension.

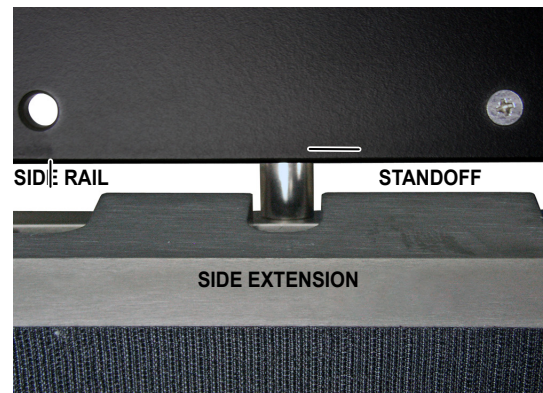

2. Pull release handle and slide side extension over rail.


WARNING
Possible Pinch/
Crush Points

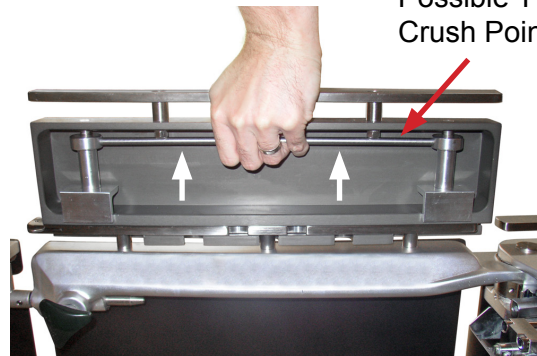

3. Release handle and verify that the side extension is firmly attached to the rail.



Width 5" (127mm)
 Length 18" (457mm)
 Weight 7.8 lbs (3.5kg)

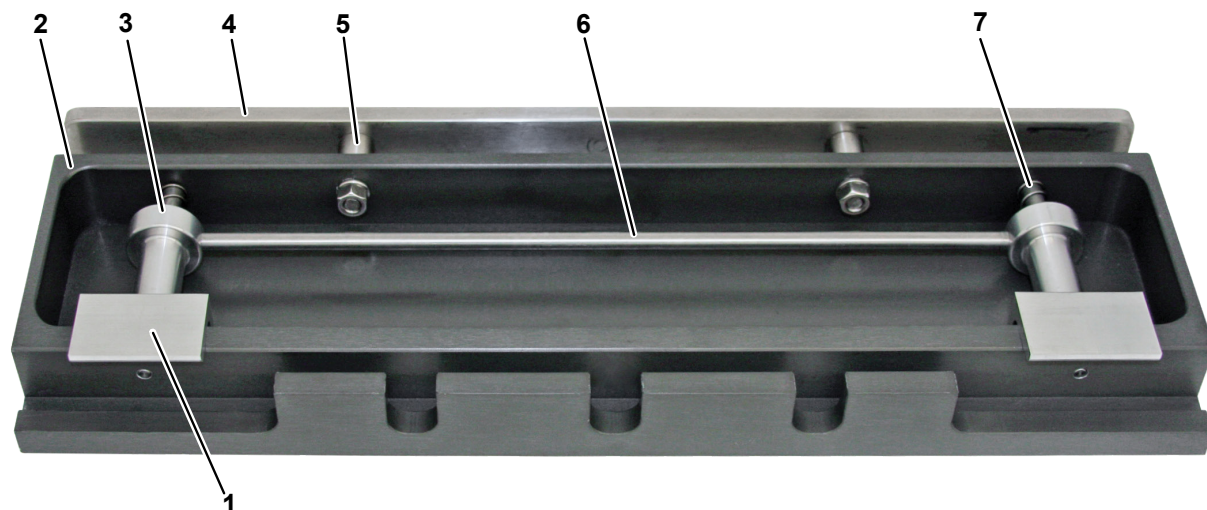

Item	Part No.	Description
1	3-060-08-5	Angle
2	3-060-08-1	Body
3	3-060-08-3	Slide Tube
4	3-060-08-9	Rail
5	3-060-08-8	Spacer
6	3-060-08-4	Handle Rod
7	3-060-08-2	Slide Bar
	3-060-08-5	Angle
Pads:		
NS	3-060-05-2P	2" Pad
NS	3-060-05-2S	2" Sof-Pad
NS	3-060-05-3S	3" Sof-Pad
NS	3-060-05-4P	4" Pad
NS	3-060-05-4S	4" Sof-Pad
NS	3-060-05-5S	5" Sof-Pad