

6" CLAMP ON RAIL - INSTRUCTIONS FOR USE

The 6" clamp on rail is designed to be used with the Skytron 40" Carbon Fiber Extension, Skytron 3000 series Imaging tables, and HyperLight leg section.

It provides a 6" long standard size side rail that can be used to mount standard accessories. The rail can be mounted on the sides or end of the 40" extension.



Specifications

Width	4.25" [108mm]
Height	4" [102mm]
Length	6" [152mm]
Product Weight	1.8 lbs [.8 Kg]

Installation



WARNING

DO NOT use the Clamp On Rail with accessories that provide traction.

1. Loosen the lock lever to provide adequate clamp clearance (Figure 1).

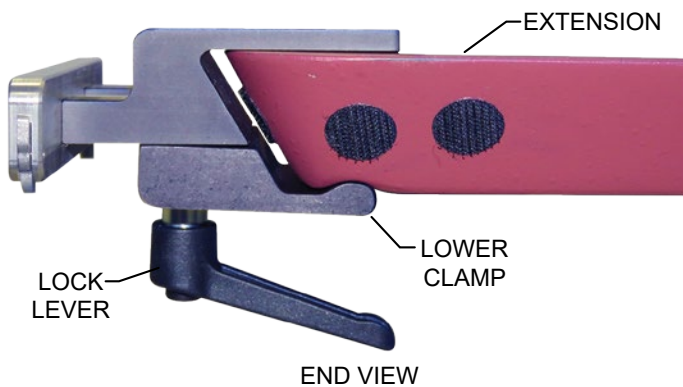


Figure 1. Rail and Extension

2. Slide the clamp over the edge of the extension and tighten the lock lever. Ensure the lower clamp is properly seated in the groove on the extension as shown.

Position the Rail

Loosen the lock lever (Figure 2), and slide the rail to the desired location on the extension. Tighten the lock lever.

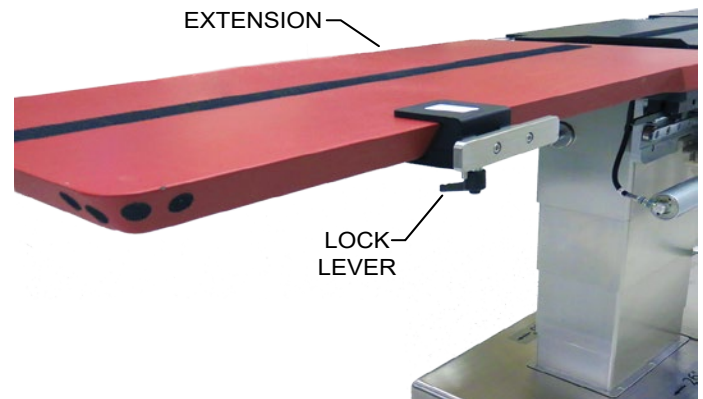


Figure 2. Rail Positioning



CAUTION

Prior to use ensure the clamp on rail is securely fastened to the extension.

Remove the Clamp On Rail

1. Loosen the lock lever to provide adequate clamp clearance (Figure 1).
2. Pull the clamp on rail straight out.

Cleaning

Always follow current AORN/EORNA Journal Guidelines to ensure proper cleaning and disinfection procedure.

Always follow product cleaning and disinfecting utilization instructions and warnings indicated by cleaning product manufacturer.

Replacement Parts

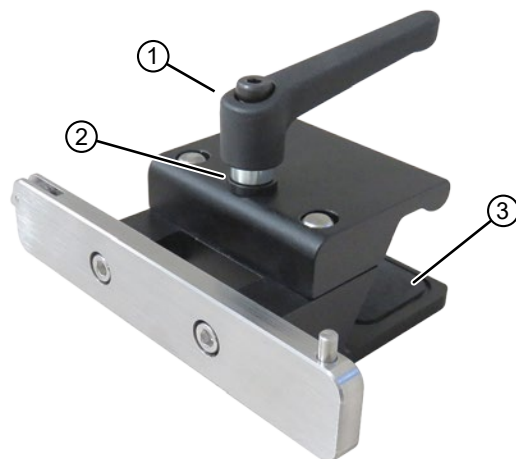


Figure 3. Clamp On Rail Components

NO.	PART NO.	DESCRIPTION	QTY,
1	2-060-04-2	LOCK LEVER	1
2	6-070-07-5	SPACER	1
3	2-060-04-P	RUBBER PAD	1

Label



Revision History

Date	Revision	Revision History
09/19/2017	0	Initial Release

Distributed by:
 US - SKYTRON
 5085 Corporate Exchange Blvd. S.E.
 Grand Rapids, MI 49512 (616) 656-2900
www.skytron.us

Manufactured by:
 Anholt Technologies
 440 Church Road
 Avondale, PA 19311
 US - UNITED STATES

 **SKYTRON®**
 5085 Corporate Exchange Blvd. S.E.
 Grand Rapids, MI 49512 • 616.656.2900