

AUXILIARY BASE CONNECTOR KITS - INSTALLATION INSTRUCTIONS

Kit Information

This document provides instructions for installing the following surgical table auxiliary base connector kits:

Kit Part Number	Used on:
D5-017-00	6300 and 3501C tables
18-443-10	6302 and 3502 tables
M001937	3503 tables

Each kit is comprised of the following components:

Kit Part Number	Components
D5-017-00	<ul style="list-style-type: none"> Connector with wiring harness and mounting hardware Gasket Adapter wiring harness
18-443-10	<ul style="list-style-type: none"> Connector with wiring harness and mounting hardware Gasket
M001937	<ul style="list-style-type: none"> Connector with wiring harness and mounting hardware Gasket

NOTICE

Adapter wiring harness (Figure 1) included in Kit# D5-017-00 is not needed for the 6300 and 3501C foot control installation).



Figure 1. Adapter Wiring Harness

Installation Instructions

Use the following procedure to install an auxiliary base connector kit:

1. Before starting, turn off the main power switch located on the table base or remove the table power cord. The battery system will turn on automatically when line power has been turned off. Depress the **BATT** button on the pendant control to turn off battery power. The pendant control backlighting will be off and no indicator lights will be lit when the table has been turned off completely.
2. Remove the service access cover and the small square cover plate on the front of the table base labeled **FOOT SWITCH** (Figure 2).
3. Install new gasket on connector. Carefully thread the wires and connector block of the auxiliary connector through the hole in the table base toward the electrical relay box. Using a pull wire will make this job easier. Attach the auxiliary connector to the table base with new screws provided.

SERVICE ACCESS COVER



Figure 2. Access Cover

4. Plug the electrical connector into connector CN8 on the relay box (Figure 3). Plug the foot control into the new base connector and check operation of the table.

5. If table operates properly, reinstall the service access cover and secure with the retaining screws.

NOTICE

Final validation of the table's operation after installation of the auxiliary base connector kit must be conducted by a SKYTRON Certified Service Technician in accordance with the specific table model's operational criteria to assure proper function and safety.

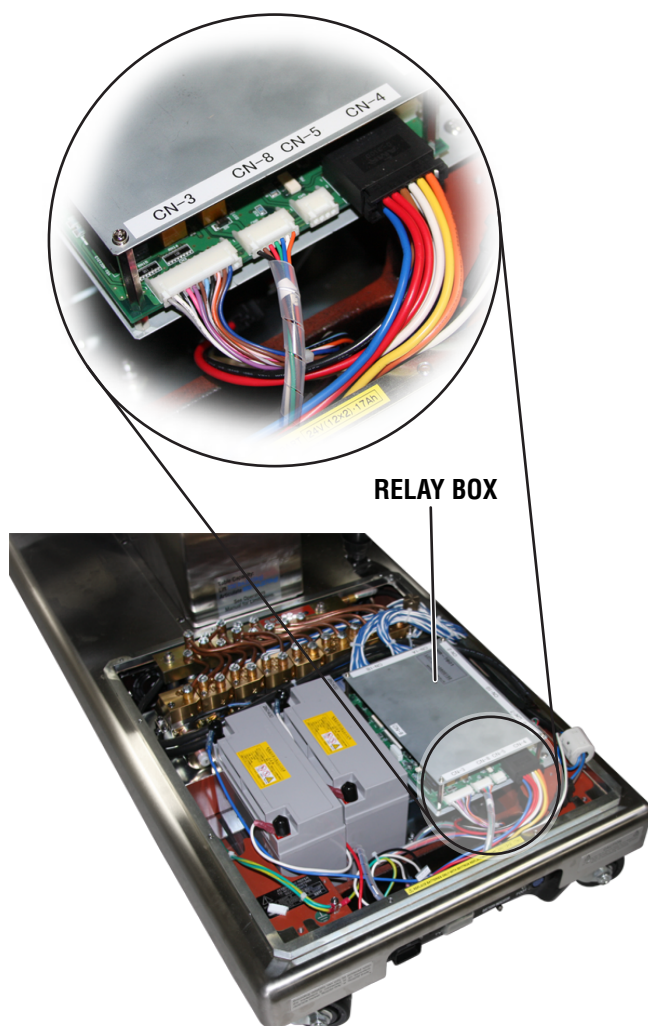


Figure 3. Connecting to Relay Box

DATE	REV	REVISION HISTORY
10/30/2017	1	Added M001937 to the part numbers and kit information tables