



5085 Corporate Exchange Blvd. S.E.
Grand Rapids, MI 49512 • 616.656.2900

PN 6-020-09

CATHETER EXTENSION - INSTRUCTIONS FOR USE

The Catheter Extension attaches to the column side of the 3003 Impulse Imaging table. The accessory serves as a surface for instrument and device support that can move in conjunction with the table as it is articulated.



Typical Dimensions

- Length 40-3/8" [1026mm]
- Width 20-1/2" [521mm]
- Height 6-7/8" [175mm]
- Max Load 10lbs [4.5kg]

Installation



WARNING

DO NOT use the catheter extension as a patient support. This accessory is intended for instrument and device support only. Injury to the patient may occur if the catheter extension is used incorrectly.

1. Tilt the catheter extension and slide the blades under the front side rail supports on the column end of the imaging table (Figure 1). The pad has been removed for clarity.

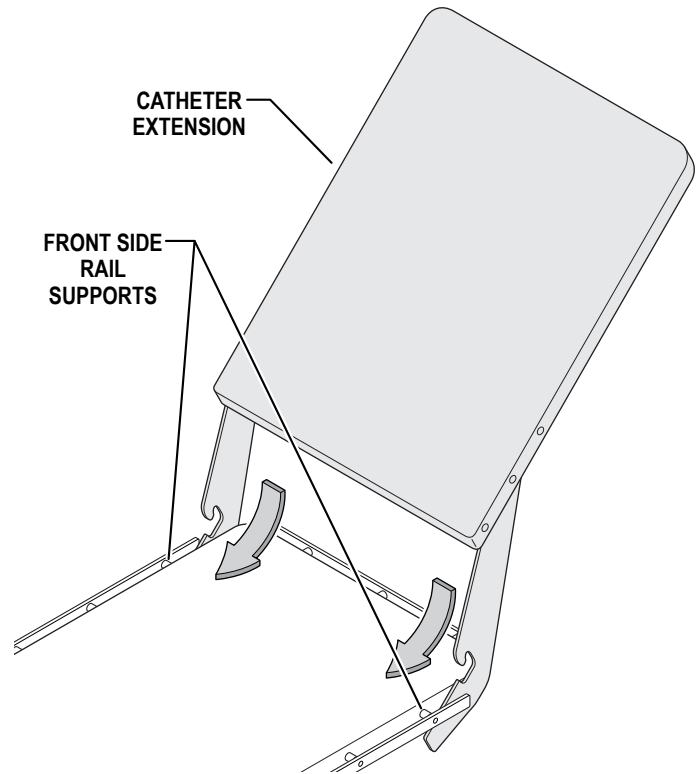


Figure 1. Align the Extension

2. Engage the hooks over the front side rail supports and rotate the extension downward to align with the imaging table. (Figure 2).

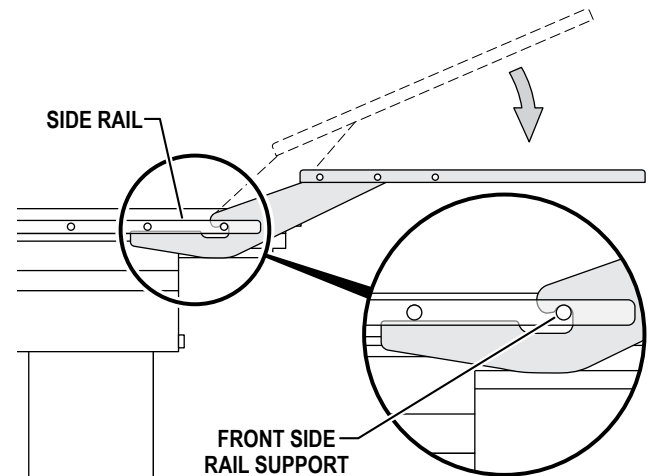


Figure 2. Attach the Extension

3. Ensure both blades on the catheter extension are under the side rail supports and secure.

Cleaning Recommendation

Follow the procedures below to clean the catheter extension between cases.



WARNING

Always follow the standards for blood-borne pathogens from the Occupational Safety and Health Administration (OSHA). Use the recommended protective clothing, gloves, masks and eye protection to clean the accessory.

1. Remove major contaminants from the accessory with disposable materials. Follow the appropriate biohazard waste disposal procedures.
2. Remove the accessory and place it on a flat surface for cleaning.

CAUTION

Thoroughly read and follow the manufacturer's directions for all cleaning fluids. **DO NOT** use cleaners containing phenolics.

3. The following cleaning and decontamination materials are known to be compatible:

- Rubbing alcohol (70% isopropyl alcohol).
 - Bleach solution (10% household-type bleach in cold, cool, or warm water).
 - General-purpose soap or detergent cleaners ("409", etc.) at their normal manufacturer-recommended strength.
4. Apply cleaning fluid liberally to the top and sides of the catheter extension.
 5. Wipe with a clean, lint-free cloth until all the moisture and cleaning fluid is removed from the accessory.
 6. Clean the bottom of the catheter extension in the same manner as describe above.
 7. Allow the accessory to dry.

Labels

The following is printed directly on the top of the catheter support.

**CATHETER EXTENSION ONLY
DO NOT USE TO SUPPORT PATIENT
OR CONTROL DEVICES IN ANY WAY
DO NOT LEAN
MAXIMUM LOAD 10 Lb. (4.5 Kg)**

Date	Revision	Revision History
10/28/2016	0	Initial Release

Distributed by:

US - SKYTRON
5085 Corporate Exchange Blvd. S.E.
Grand Rapids, MI 49512 (616) 656-2900
www.skytron.us

Manufactured by:



CFI Medical Solutions
14241 Fenton Road
Fenton, MI 48430
US-UNITED STATES

The base language for this document is ENGLISH. Any translations must be from the base language document.
Printed copies are not controlled documents.

Although current at the time of publication, SKYTRON'S policy of continuous development makes this document subject to change without notice. If a current document is required, contact your local SKYTRON representative or contact SKYTRON directly at the distribution addresses listed above.

 **SKYTRON®**
5085 Corporate Exchange Blvd. S.E.
Grand Rapids, MI 49512 • 616.656.2900 • FAX 616.656.2906