



5085 Corporate Exchange Blvd. S.E.
Grand Rapids, MI 49512 • 616.656.2900

EZ GRIP BAR CLAMP

P.N. 6-030-05-1

INSTRUCTIONS FOR USE:

Place clamp over rail at a 45° angle over rail cut outs.
(Figure 1)

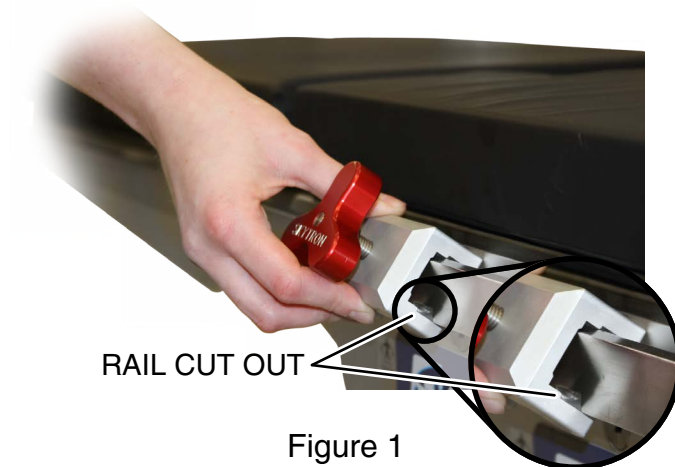


Figure 1

Lower the clamp so that it is parallel to the rail and slide it into place. Insert accessory and tighten the handle. (Figure 2)



Figure 2



Do not position the clamp for use over the rail cut outs. Damage to the clamp or instability of the accessory may occur.



Do not position the clamp for use in between table top sections.

The SKYTRON EZ Grip Bar Clamp is designed for use with standard bar stock accessories measuring 1" x 1/4" (25.4mm x 6.4mm).

The SKYTRON EZ Grip Bar Clamp works great with the SKYTRON Levitator Sirrups.

P.N. 4-090-03-1

P.N. 4-090-03-1HD

Replacement Parts:

None

Typical Dimensions:

Length	3" (76mm)
Width	1 3/4" (44mm)
Height	2 5/8" (67mm)

Weight:

.50lb (.23kg)

Authorized Distributors:

U.S. - SKYTRON

5085 Corporate Exchange Blvd. S.E.
Grand Rapids, MI 49512

Manufactured by:

B.G.I. Tooling, Co.
806 B. Gillette Rd.
Colville, WA 99114

