




PN 6-030-05-HD
PN 6-040-07-HD

HD SUPPORT RODS - INSTRUCTIONS FOR USE

Skytron HD Support Rods should be used to stabilize unevenly distributed patient weight or when using specific top positioning on Skytron top rotation tables.

The support rod must be positioned on the side of the table with the most weight, and should never be attached to the tables head section.

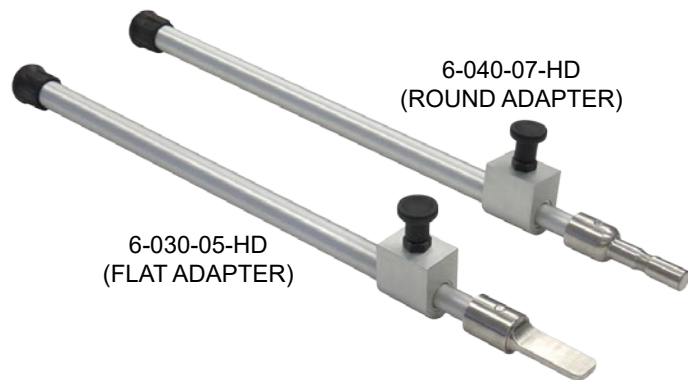

Specifications

6-030-05-HD (flat adapter)

Retracted Length: 26" [660mm]

Extended Length: 44" [1117mm]

Weight: 2.85 lbs [1.3kg]

Use With Clamp: 6-030-05-1



6-040-07-HD (round adapter)

Retracted Length: 26" [660mm]

Extended Length: 44" [1117mm]

Weight: 2.85 lbs [1.3kg]

Use With Clamp: 6-040-10



Installation

NOTICE

Although support rods 6-030-05-HD and 6-040-07-HD require different side rail clamps, installation and adjustment are the same.



WARNING

DO NOT attach support rods to the head section of the table!

Top Rotation Tables:

If the centerline of the table top is positioned within the red range (Figure 1), the weight capacity is consistent with the overall lifting and articulating capacity of the table. The use of a support rod is not necessary.

If the centerline of table top is positioned within the blue range, the weight capacity is consistent with the overall lifting capacity of the table. A heavy duty support rod **MUST BE USED** on the back section. **The table cannot be articulated when the support rod is used.**

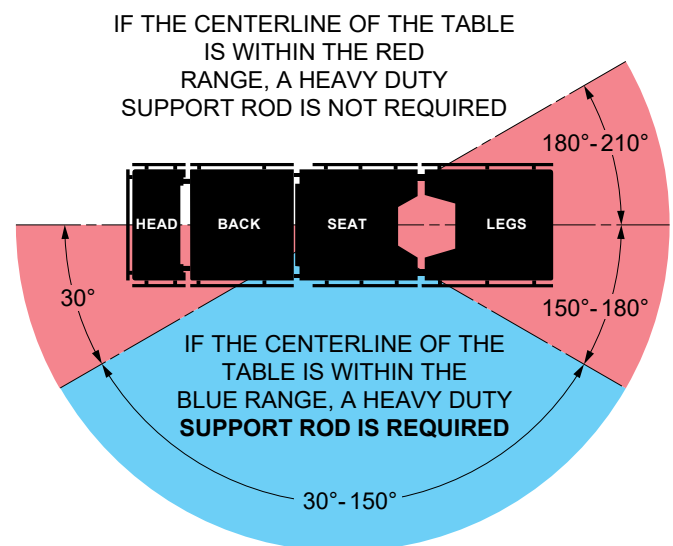


Figure 1. Rotation Support Zone

1. Attach the bar clamp to the back section side rail (Figure 2).

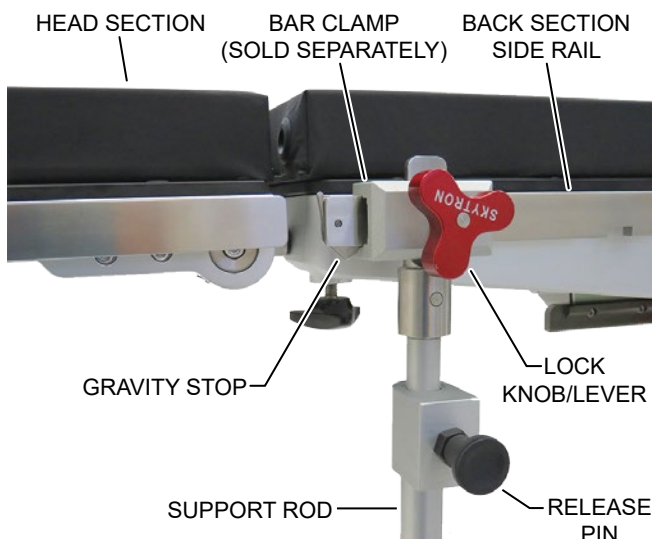


Figure 2. Support Rod Attachment

2. Slide the bar clamp toward the head section until it touches the gravity stop.
3. Insert the retracted support rod into the bar clamp and tighten the lock knob/handle.
4. Pull the release pin and lower the support rod as close to the floor as possible. Release the lock pin and ensure it snaps into place .
5. Gently lower the table until the support rod is snug against the floor (Figure 3).



Figure 3. Installed Support Rod

Removal

1. Raise the table slightly to remove tension on the support rod.
2. Pull the release pin and raise the support rod as far as possible.
3. Loosen the lock knob/handle and remove the support rod from the bar clamp.
4. Remove the bar clamp from the side rail.

Cleaning

Always follow current AORN/EORNA Journal Guidelines to ensure proper cleaning and disinfection procedure.


Always follow product cleaning and disinfecting utilization instructions and warnings indicated by cleaning product manufacturer.

Date	Revision	Revision History
09/14/2017	0	Initial Release

Distributed by:

US - SKYTRON
5085 Corporate Exchange Blvd. S.E.
Grand Rapids, MI 49512 (616) 656-2900
www.skytron.us

Manufactured by:

 Freedom Manufacturing, Inc
5320 6 Mile C. NW
Comstock Park, MI 49321
www.freedom-mfg.com

 **SKYTRON®**
5085 Corporate Exchange Blvd. S.E.
Grand Rapids, MI 49512 • 616.656.2900 • FAX 616.656.2906