

Introduction

The light weight armboard attaches to the side rails of SKYTRON surgical tables to support a patient's arm(s) during surgical procedures. It weights only 3.9 lbs. (1.77 kg) but will support the arms of a patient weighing up to 1200 lbs (544 kg).

Installation

a. With the armboard positioned at a slight angle, place the top clamp of the mounting block assembly over the table side rail.

b. Lower the armboard downward until level with the table top. The spring loaded lower rail lock clamp should lock the armboard in place.

c. Ensure that the armboard is securely fastened to the table side rail.

d. Install the desired arm pad (velcro side down). The following arm pads are available for the light weight armboard:

Part Number	Description
2-010-07-2WS	2" Welded Seam Pad
2-010-07-3WS	3" Welded Seam Pad
2-010-07-4WS	4" Welded Seam Pad
2-010-07-5WS	5" Welded Seam Pad

Adjustment

To adduct and abduct the armboard, pull the rotator trigger, position the armboard, and release the trigger (Figure 1).

Page 1



Figure 1. Rotator Trigger

Before Use

- Be sure that the armboard is securely attached to the table side rail.
- DO NOT locate the armboard next to or partially over the clark socket cutout on the table side rail.
- Be sure the armboard is positioned to avoid any pinch points before articulating the table.

Removal

a. Remove the arm pad from the arm board.

b. Adjust the armboard so it is positioned approximately 90° to the table side rail.

c. Depress the rail lock lever and lift the armboard up to release it from the table side rail (Figure 2).



Figure 2. Rail Lock

Cleaning and Disinfecting



Always follow OSHA blood-borne pathogens standards for protective clothing, including gloves, masks, and eye protection when cleaning this accessory.

NOTICE

Always follow current AORN Journal Guidelines to ensure proper cleaning and disinfection procedures.

Always follow product cleaning and disinfecting instructions and warnings provided by the cleaning product manufacturer.

Use the same cleaning and disinfecting procedures as recommended in your table's Owner's manual to clean and disinfect this accessory between cases.


Service

Maintenance can be performed by trained maintenance personnel using SKYTRON authorized replacement parts and service techniques. Service instructions and parts are available from SKYTRON.

To obtain service instructions, replacement parts, or factory service, contact your local SKYTRON representative.



Replacement Parts

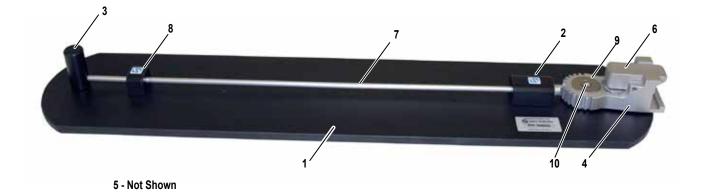

ltem	Part Number	Description	Quantity
1	2-010-07-1	ARMBOARD, REGULAR PHENOLIC	1
2	2-010-07-2	ARMBOARD SLIDE BLOCK # 1	1
3	2-010-07-3	ARMBOARD TRIGGER ROTATOR	1
4	2-010-07-5	ARMBOARD MOUNTNG BLOCK	1
5	2-010-07-6	ARMBOARD BLOCK HUB	1
6	2-010-07-7	ARMBOARD RAIL LOCK	1
7	2-010-07-8	ARMBOARD INDEXING BAR	1
8	2-010-07-9	ARMBOARD SLIDE BLOCK # 2	1
9	2-010-07-10	WASHER, S.S., 5/16"	1
10	2-010-07-11	SCREW, S.S., BUTTON HEAD SOCKET, 1/4-20 X 5/8"	1



Authorized Distributors: U.S. - SKYTRON 5085 Corporate Exchange Blvd. S.E. Grand Rapids, MI 49512

EUROPE - SKYTRON EUROPE BV

FLORESSTRAAT 52 8022 AD ZWOLLE THE NETHERLANDS EC REP

CE

Manufactured by: SchureMed 452 Randolph Street Abington, MA 02351

