

LOWER BACK SECTION - INSTRUCTIONS FOR USE

The Lower Back Section accessory provides options to configure and shorten the surgical table. This accessory allows the anesthesiologist better access to the patient when the patient is positioned toward the foot end of the surgical table.



1. Lower Back Section

Length	9.5 inches
	508 millimeters
Width	20 inches
	241 millimeters
Weight	7 pounds
	3 kilograms
Weight Limit	800 pounds
	363 kilograms

Installation

Use the instructions below to install the lower back section accessory.

1. Remove the head section of the table.
 - a.) Remove the pad.
 - b.) Rotate the two locking knobs under the head section to loosen them.
 - c.) Pull the head section straight out.



WARNING

The head section is heavy. Safely handle the leg section during removal to prevent injury.

2. Remove the back section of the table.
 - a.) Make sure the surgical table top is level.
 - b.) Simultaneously squeeze the two release levers on the bottom of the back section and pull the leg section off the attaching pins.



WARNING

The back section is heavy. Safely handle the leg section during removal to prevent injury.

3. Slide the lower back section onto the pins on the surgical table.
4. Rotate the two mounting clamp handles under the lower back section accessory to tighten the clamps on the pins.



2. Tighten the Clamps

5. Cover the lower back section with the head section pad.
6. Cover the seat section of the surgical table with the leg section pad.



This product supports the head of the patient only. Do NOT use this accessory for a pelvic tilt seat.

Cleaning Recommendation



Always follow the current Association of periOperative Registered Nurses (AORN) Journal Guidelines to make sure the proper cleaning and disinfection procedure is followed.

Follow the procedures below to clean the lower back section between cases.



WARNING

Always follow the standards for blood-borne pathogens from the Occupational Safety and Health Administration (OSHA). Use the recommended protective clothing, gloves, masks and eye protection to clean the accessory.

1. Remove major contaminants from the accessory with disposable materials. Follow the appropriate biohazard waste disposal procedures.
2. Remove the accessory and place it on a flat surface for cleaning.

CAUTION

Thoroughly read and follow the manufacturer's directions for all cleaning fluids. DO NOT use cleaners containing phenolics.

3. Use a ready-to-use detergent diluted as required by the manufacturer. Make sure the active ingredients of the product are compatible with the materials used in the lower back section.

4. Apply cleaning fluid liberally to the top and sides of each lower back section.
5. Wipe with a clean, lint-free cloth.
6. Wipe until all the moisture and cleaning fluid is removed from the accessory.
7. Apply cleaning fluid liberally to the bottom of the each lower back section.
8. Wipe with a clean, lint-free cloth.
9. Wipe until all the moisture and cleaning fluid is removed from the bottom of the accessory.
10. Allow the accessory to dry.

Disinfection

The following antiseptic solutions are approved for use on the lower back section:

- Sodium Hypochlorite 6% diluted to 0.1% (halogen containing compound)
- HypoAlcohol (iodine color removing agent)
- Chlorhexidine (chlorhexidine gluconate 0.5%)
- Benzalkonium Chloride (invert soap 10%)
- Povidone Iodine
- Ethanol 80%
- Oxydol (hydrogen peroxide)
- Isopropyl Alcohol (IPA) 99.5%

Use the following steps to disinfect the lower back section.

Remove all pads from the lower back section, if applicable.

Apply disinfectant to a clean, lint-free cloth.

Wipe the top, sides and bottom of the accessory with the cloth.



Make sure there is enough disinfectant on the cloth so the cloth does not dry out before the entire accessory is wiped and disinfected.

REVISION HISTORY

Date	Revision	Revision History
03/09/2016	0	Initial Release
06/03/2016	1	Corrected part number from 4-030-13 to 3-040-24
09/05/2017	2	Added Emergo Group
09/28/2017	3	Updated Emergo address


Distributed by:

SKYTRON
5085 Corporate Exchange Blvd. S.E.
Grand Rapids, MI 49512
616-656-2900
www.skytron.us

Manufactured by:



SchureMed
452 Randolph Street
Abington, MA 02351
888-724-8763
www.schuremed.com

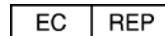

Emergo Europe B.V.
Prinsessegracht 20, 2514 AP
The Hague, The Netherlands