



PN 7-010-02-B

NEURO ADAPTER - INSTRUCTIONS FOR USE

The Skytron 7-010-02-B Neuro Adapter is designed to easily attach to Skytron General purpose tables with removable head sections.

It provides a compatible mounting location for Mayfield type base units.



Typical Dimensions

Width: 17" [432mm]

Length: 7" [178mm]

Height: 4.25" [108mm]

Weight: 4 lbs [1.8kg]

Head Section Removal

1. Remove the existing head section by loosening the two locking knobs and pulling it straight out of the back section (Figure 1).

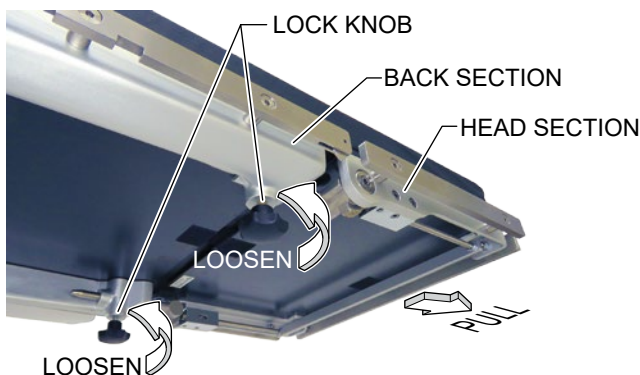


Figure 1. Remove Head Section

Installation



WARNING

DO NOT position slide top tables toward the leg end of the table when using the Neuro Adapter. Doing so can result in inadvertent removal of the adapter or cause damage to the table, adapter or third party headrest.

1. Center the table or position it toward the head end.
2. Orient the adapter with the lock knobs pointing towards the floor.
3. Align the adapter pins with the holes in the back section (Figure 2).



Figure 2. Neuro Adapter

4. Slide the adapter in until it bottoms out against the table back section.
5. Tighten both back section lock knobs and ensure the adapter is secure (Figure 3).

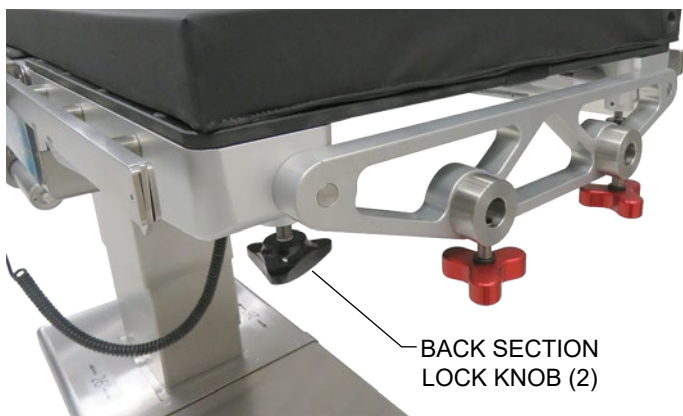


Figure 3. Installed Adapter

Component Attachment

1. Align the attachment pins of the desired Mayfield component you wish to use, with the holes in the Neuro adapter (Figure 4).

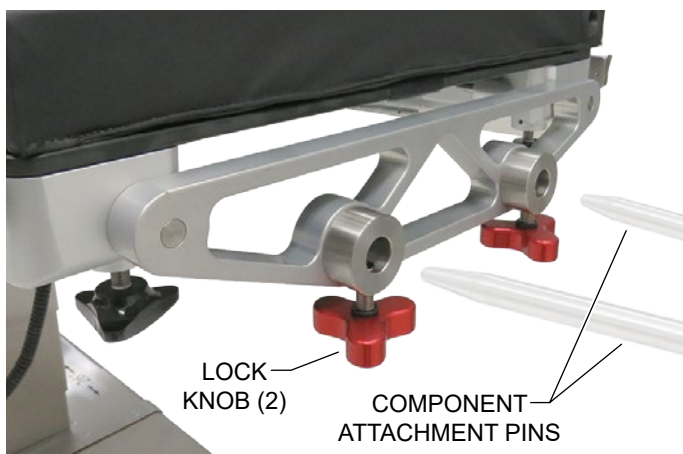


Figure 4. Component Attachment


2. Slide the Mayfield component into the adapter and tighten both lock knobs.

Cleaning

Always follow current AORN/EORNA Journal Guidelines to ensure proper cleaning and disinfection procedure.

Always follow product cleaning and disinfecting utilization instructions and warnings indicated by cleaning product manufacturer.

Replacement Parts

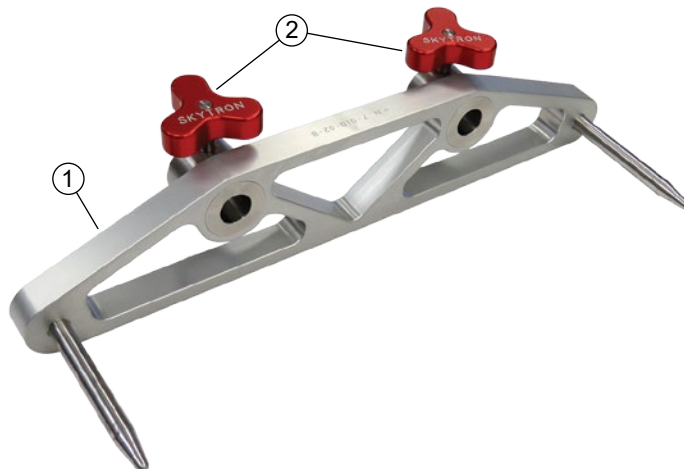

NO.	PART NO.	DESCRIPTION	QTY,
1	7-010-02-B	NEURO ADAPTER (INCLUDES LOCK KNOBS)	1
2	4-030-12-7R	LOCKING KNOB WITH SPACER AND BUTTON	2

Date	Revision	Revision History
09/19/2017	0	Initial Release

Distributed by:

US - SKYTRON
5085 Corporate Exchange Blvd. S.E.
Grand Rapids, MI 49512 (616) 656-2900
www.skytron.us

Manufactured by:

 BGI Tooling, Co
806B. Gillette Rd.
Colville, WA 99114
www.bgitool.com

 **SKYTRON®**
5085 Corporate Exchange Blvd. S.E.
Grand Rapids, MI 49512 • 616.656.2900 • FAX 616.656.2906