



PN 4-030-42

PEDIATRIC TABLE EXTENSION - INSTRUCTIONS FOR USE

The Skytron Pediatric Table Extension is designed to fit the 6700, 6600, 6500, 6300, 3600, and 3500 series of Skytron general purpose surgical tables. It is intended to provide better surgical access to smaller patients as well as providing great imaging with a carbon fiber constructions and customization with its unique detachable side rails.



WARNING

DO NOT exceed 100 pounds on the pediatric table extension.



CAUTION

This product should only be used by trained personnel. Personnel must have received clinical in-service instructions.

Technical Data:

Patient Weight Limit: 100 pounds (34.35 kg)

Weight with Clamps: 24.6 lbs (11.16 kg)

Length: 40" (1016 mm)

Width: 15" (381 mm)



Install the Pediatric Table Extension

1. Remove the table leg section pad set.
2. Remove the x-ray top from the table, if applicable.
3. Adjust the table top to a safe and comfortable height to remove the leg section. Raise or lower the table top as needed.



CAUTION

Follow the requirements for the positioning and the articulation of the specific table used with the pediatric table extension.




CAUTION

Only personnel trained in the use of this product with the specific surgical table may use this accessory. Personnel must have received clinical in-service use instructions for the specific surgical table.

4. Press the two release levers on the leg section of the table.
5. Pull the leg section out from the leg pins while the two release levers are pressed.
6. Place the pediatric table extension on the leg pins on the table.
7. Push the pediatric table extension until it rests against the leg pins AND the pins click into place.

8. Pull the pediatric table extension to make sure it is secure.
9. Align the 8" clamps on the rails with the groove in the bottom of the extension, where desired.

CLAMP (shown from the bottom)

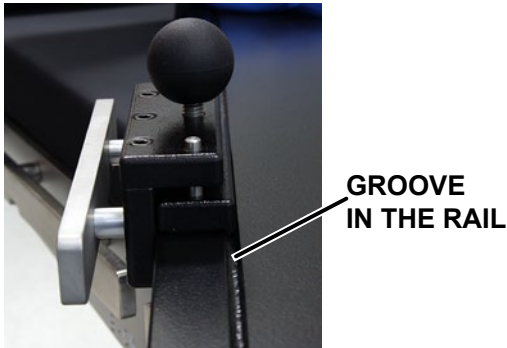

10. Rotate the ball locking knob clockwise to tighten each clamp in place.

Remove the Pediatric Table Extension

1. Rotate the ball locking knob counter-clockwise to loosen the clamp.
2. Remove both clamps.
3. Press the two release levers on the pediatric table extension.
4. Pull the pediatric table extension out from the leg pins while the two release levers are pressed.

Cleaning and Maintenance

Each cushion should be cleaned on a routine basis as determined by hospital policies and decontamination practices.

- Remove the pad sets from the table and clean and disinfect them between cases.
- Examine the pads for signs of wear or damage. Determine if the seal on the pad is compromised.
- Wash the pads and armboard with one of the two options below. **Do not immerse the cushions.**
 - Wash the pads and armboard with a generous application of neutral soap suds and lukewarm water. Rinse with water and dry.
 - Use a diluted bleach solution of 10-parts water to 1-part bleach to wash the pads and armboard. Wipe the pads and armboard dry with a clean cloth. **Bleach MUST be diluted to the 10:1 ratio.**

- Disinfect the cushions and armboard with one of the two methods below:

- Use Lysol Brand III I.C. Disinfectant Aerosol Spray by Reckitt Benckiser, Inc., Montvale, NJ, per manufacturer's instructions. Reference: Reckitt Benckiser UPC # 36241-95029-3 to disinfect the cushions and armboard.
- Use a diluted bleach solution of 10-parts water to 1-part bleach to wash the pads and armboard. Wipe the pads and armboards dry with a clean cloth. **Bleach MUST be diluted to the 10:1 ratio.**




WARNING

When the cushions can no longer be used due to wear, dispose of them properly. Pads should be treated as hazardous waste.

Replacement Parts


Part Number	Part Description	Quantity
4-030-42-1	Carbon Fiber Top P.T.E.	1
4-030-42-2	Pin Block P.T.E.	2
4-030-42-3	Lock Block P.T.E.	2
4-030-42-4	Lever P.T.E.	2
4-030-42-5	Screw, 10-32x1 P.T.E.	8
4-030-42-6	Screw 1/4-20x1.25 P.T.E.	12
4-030-42-7	Spring P.T.E.	2
4-030-42-8	Dowel Pin 1/4x1	2

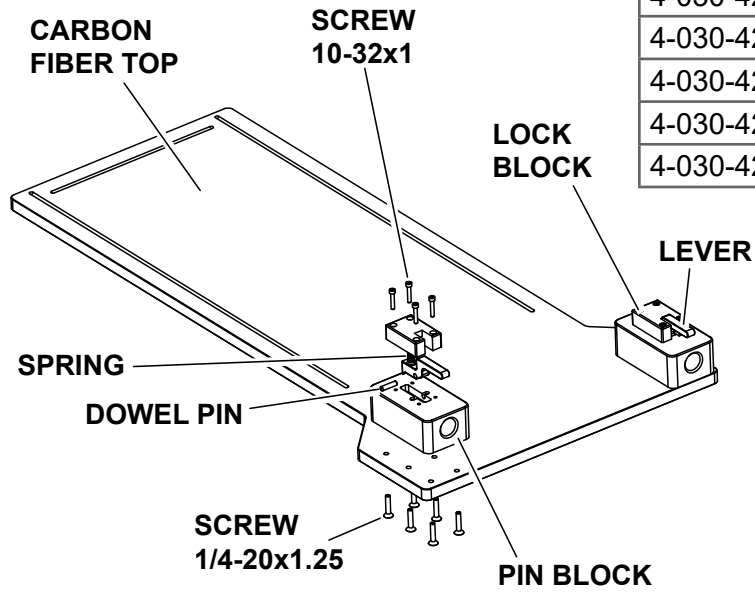

Part Number	Part Description	Quantity
4-030-42-2WS	2" Welded Seam Pad P.T.E.	1
4-030-42-3WS	3" Welded Seam Pad P.T.E.	1
4-030-42-4WS	4" Welded Seam Pad P.T.E.	1
4-030-42-5WS	5" Welded Seam Pad P.T.E.	1
4-030-42-9	Clamp on Rail, pair	1

Labels


Weight Capacity Label



**Pediatric Table
Extension 4-030-42**
Patient Weight Limitations


Model 3500 & 3600 Series

Used in place of leg section limited to a **100 lb. [45kg]** patient.





Model 6300 & 6700 Series

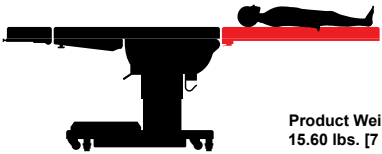Used in place of leg section limited to a **100 lb. [45kg]** patient.



Product Weight
15.60 lbs. [7 kg]

Product Label

REF 32470 <small>CATALOG NUMBER</small>	SN 123456-001 <small>SERIAL NUMBER</small>	<div style="display: flex; justify-content: space-between;"> <div style="width: 45%;">  <p><small>MANUFACTURED BY:</small> CFI Medical 14241 Fenton Road Fenton, MI 48430 USA www.cfimedical.com</p> </div> <div style="width: 45%;">  <p><small>MANUFACTURED DATE</small> 01/2009</p> </div> </div> <p style="text-align: center; font-size: small;">32470-310-000</p>
---	--	---

Revision History

Date	Revision	Revision History
07/07/2016	0	Initial release.
09/28/2016	1	Add the weight capacity label
10/13/2017	2	Add product label

Distributed by:

SKYTRON

5085 Corporate Exchange Blvd. S.E.

Grand Rapids, MI 49512

616-656-2900

www.skytron.us

Manufactured by:



Contour Fabricators Inc.

14241 Fenton Rd

Fenton, MI 48430

810-750-5300

cfimedical.com


SKYTRON®
5085 Corporate Exchange Blvd. S.E.
Grand Rapids, MI 49512 • 616.656.2900