



5085 Corporate Exchange Blvd. S.E.
Grand Rapids, MI 49512 • 616.656.2900

PN 2-020-01-1

RAISED ARMBOARD - INSTRUCTIONS FOR USE

The Raised Armboard provides support to the unaffected arm during surgery. This accessory is used when the patient is in a beach chair position.



A clamp is not included with the raised armboard.

Length	16 inches
	406 millimeters
Width	5 inches
	127 millimeters
Weight	4.2 pounds
	1.92 kilograms



1. Raised Armboard on the Surgical Table

Installation with the Recommended Schure Clamp



A clamp is not included with the raised armboard. The Schure Socket XL clamp installation is described below with the installation of the raised armboard. The Schure Socket XL clamp is the recommended clamp for this accessory.

1. Hold the Schure Socket XL clamp at a slight angle with the rear of the clamp above the side rail on the surgical table.
2. Lower the clamp to the horizontal position.



2. Install the Schure Socket XL Clamp

3. Insert the mounting bar into the clamp.
4. Move the mounting bar up or down to adjust the height of the raised armboard.

5. Tighten the handle on the clamp to lock the clamp and mounting bar to the side rail.
6. Rotate the locking handle counterclockwise to release the armboard.



When the locking handle is released, the armboard has a wide range of motion for adjustment.

7. Adjust the armboard, as desired.
8. Rotate the locking handle clockwise to lock the armboard into position.
9. Complete the sterilization and draping procedures per the hospital requirements.
10. Place the patient's unaffected arm on the armboard.
11. Wrap the straps around the armboard and the patient's arm.
12. Secure the velcro on the straps.

Preventive Maintenance

Complete the preventive maintenance identified below. Regular preventive maintenance makes sure the accessory operates correctly. Preventive maintenance should only be completed by qualified and trained personnel from Skytron.

- Complete a general, visual examination during cleaning. Check for:
 - Loose bolts
 - Loose parts
 - Cracks

Complete the necessary repairs or replace the accessory.

- Complete a periodic inspection of the following:
 - Operation of the clamps
 - Operation of the locking handle
 - Surface of the armboard
 - Operation of the straps

Cleaning Recommendation



Always follow the current Association of periOperative Registered Nurses (AORN) Journal Guidelines to make sure the proper cleaning and disinfection procedure is followed.



WARNING

Always follow the standards for blood-borne pathogens from the Occupational Safety and Health Administration (OSHA). Use the recommended protective clothing, gloves, masks and eye protection to clean the table extension.

To clean the raised armboard, use the procedures in place to clean the surgical table. Disinfect this surgical table accessory between cases.

For soils and stains:

- Wash the accessory with a large amount of neutral soap suds and lukewarm water. Rinse with water. Dry the table extension. **Do NOT** immerse the table extension in water.

OR

- Wash the accessory with a diluted bleach solution. The bleach must be 10 parts water to 1 part bleach. Wipe the accessory dry with a clean cloth. **Do NOT** immerse the table extension in water. Bleach **MUST** be diluted to the 10:1 ratio.

To disinfect:

- Use Lysol I.C. Disinfectant Spray by National Laboratories, Montvale, NJ per manufacturer's instructions. Reference: National Laboratories Item # 95029-000

OR

- Wash the accessory with a diluted bleach solution. The bleach must be 10 parts water to 1 part bleach. Wipe the accessory dry with a clean cloth. **Do NOT** immerse the table extension in water. Bleach **MUST** be diluted to the 10:1 ratio.

When the accessory needs to be disposed of, treat it as hazardous waste. Dispose of the accessory properly.

After the raised armboard is clean, store the accessory and clamps according to the standard procedure for the hospital. This will maintain the cleanliness.

Inspect before and after each use.

Replacement Parts



Item	Quantity	Part Number	Description
1		6-040-11	SCHURE SOCKET XL, each
2		2-020-01-2	RAISED ARMBOARD STRAP, each
3		2-020-01-3	ARMREST SECTION, each

REVISION HISTORY

Date	Revision	Revision History
04/25/2016	0	Initial Release

NOTES

Distributed by:

SKYTRON

5085 Corporate Exchange Blvd. S.E.

Grand Rapids, MI 49512

616-656-2900

www.skytron.us

Manufactured by:



Mizuho Corporation

3-30-13 Hongo, Bunkyo-ku

Tokyo 113-0033

JAPAN

www.mizuho.co.jp/english/index.html



5085 Corporate Exchange Blvd. S.E.
Grand Rapids, MI 49512 • 616.656.2900