

SCHURE SOCKET XL - INSTRUCTIONS FOR USE

The SKYTRON Schure Socket XL attaches standard flat 1" x 3/8" (25.4mm x 9.5mm) and round 5/8" (15.9mm) bar accessories to a table's side rail. It can be easily attached and removed from the table side rail with one hand.

Installation

1. Hold the Schure Socket XL at a slight angle with the rear of the clamp above the side rail and then lower the clamp to the horizontal position.



2. Position the accessory inside the clamp.
3. Tighten the handle to lock the Schure Socket XL and the accessory to the side rail.



Replacement Parts:

Handle (P.N. 6-040-11-1)

Typical Dimensions:

Width: 1-1/2" (63mm)

Length (w/ Handle): 4-1/2" (114mm)

Weight:

16.8 oz. (.48 kg)

DATE	REV	REVISION HISTORY
09/05/2017	3	Added Emergo Europe
09/27/2017	4	Updated Emergo Europ address

Manufactured by:



SchureMed
452 Randolph Street
Abington, MA 02351

Distributed by:

US - SKYTRON
5085 Corporate Exchange Blvd. S.E.
Grand Rapids, MI 49512 (616) 656-2900
www.skytron.us

EC	REP	Emergo Europe B.V. Prinsessegracht 20, 2514 AP The Hague, The Netherlands
----	-----	---