

URO TRAY BRACKET INSTALLATION - INSTRUCTIONS FOR USE

The Uro Tray is an accessory used on Skytron 3500 and 3600 series surgical tables. This tool is used for cleaning gynecological, urological, and other operations. A fine mesh screen is provided to prevent washing and body solutions from splashing, and to check for foreign matter.



The Uro Tray Bracket is oriented differently on the 3500 and 3600 tables.

CAUTION

Damage to the shrouds will result from improper installation! To ensure proper Uro Tray operation, refer to the figures below for proper bracket orientation.

1-1. Remove Leg Section

1. Remove the pad set.
2. Remove the x-ray top from the table.
3. Adjust the height of the table top to a safe and comfortable height required to remove the leg section.
4. Simultaneously squeeze the two release levers on the bottom of the leg section and pull the leg section off the attaching pins.



WARNING

The leg section weighs approximately 20 pounds (9.07 kilograms). Safely handle the leg section during removal.

1-2. Model 3500 Bracket Installation

1. Orient the thicker portion of the bracket above the screw slots facing up (Figure 1).

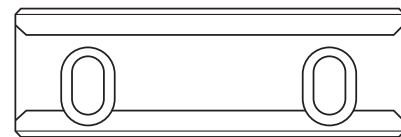
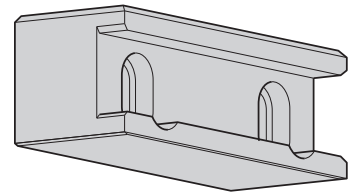


Figure 1. 3500 Bracket Orientation

2. Using the existing tapped holes, attach the Uro Bracket to the side frame with two Allen Bolts (Figure 2).

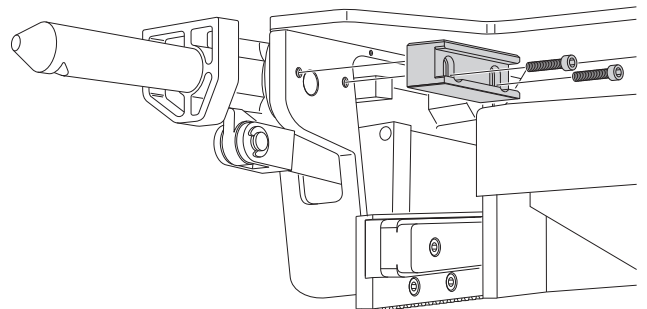


Figure 2. 3500 Bracket Installation

3. Repeat steps one and two to install the second bracket on the opposite side.

1-3. Model 3600 Bracket Installation

1. Orient the thinner portion of the bracket above the screw slots facing up (Figure 3).

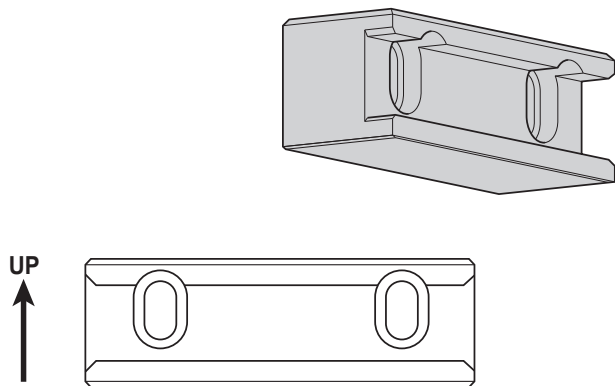


Figure 3. 3600 Bracket Orientation

2. Using the existing tapped holes, attach the Uro Bracket to the side frame with two Allen Bolts (Figure 4).

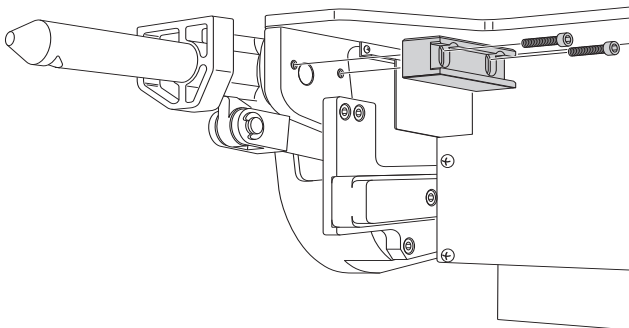


Figure 4. 3600 Bracket Installation

3. Repeat steps one and two to Install the second bracket on the opposite side.

1-4. Install the Uro Tray

1. Place the Debris Screen into the Uro Tray with the D ring facing up (Figure 5).

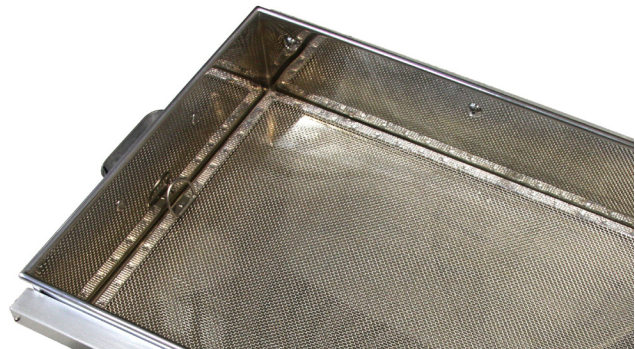


Figure 5. Debris Screen Installed

2. Insert both legs of the Uro Tray into the brackets and slide forward until it stops (Figure 6).

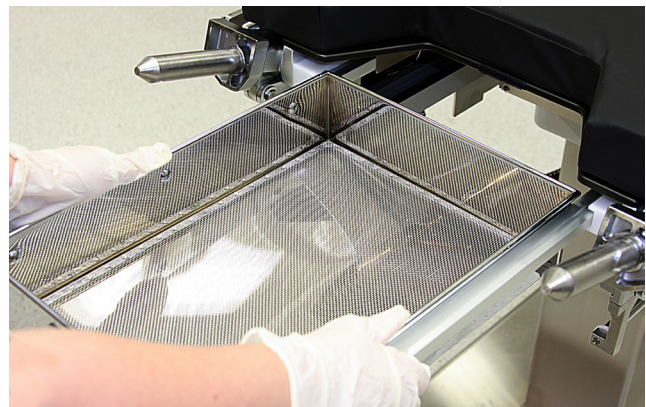


Figure 6. Uro Tray Installed

1-5. Install Leg Section

1. Reinstall the Leg Section on the attaching pins and push in until it rests against the pin collar and clicks into place.
2. Pull on the Leg Section to verify it is secure.
3. Reinstall the pad.

1-6. Uro Tray Operation



CAUTION

When removing the Uro Tray use extreme caution to prevent accidental injury or exposure to contaminants that may be present.

1. When removing the Uro Tray, It's important to grasp the handle from above and pull straight out until resistance is met. (Figure 7).

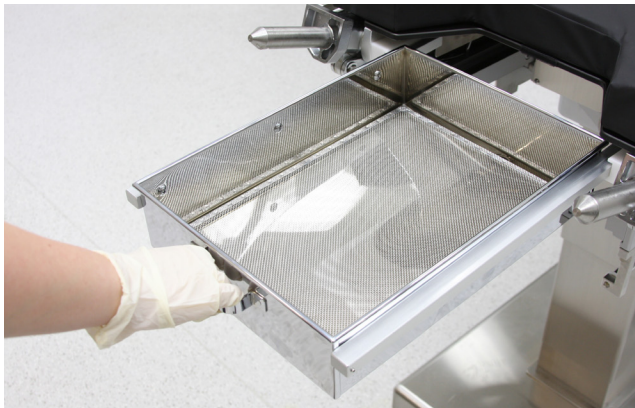


Figure 7. Uro Tray Removal

2. Using two hands, tilt the tray up slightly and pull out until the tray clears the edge of the table (Figure 8).



Figure 8. Uro Tray Installed

1-7. Cleaning Recommendations



Always follow the current Association of periOperative Registered Nurses (AORN) Journal Guidelines to make sure the proper cleaning and disinfection procedure is followed.



WARNING

Always follow the standards for blood-borne pathogens from the Occupational Safety and Health Administration (OSHA). Use the recommended protective clothing, gloves, masks and eye protection when cleaning the Uro Tray.

1. Use disposable cleaning materials to remove major contaminants. Follow the appropriate procedures to dispose of biohazardous waste.

Date	Revision	Revision History
5/2006	0	Initial Release
10/14/2014	1	Captions for the trays changed
01/25/2016	2	Complete document refresh.

Distributed by:

US - SKYTRON
5085 Corporate Exchange Blvd. S.E.
Grand Rapids, MI 49512 (616) 656-2900
www.skytron.us

Manufactured by:



Mizuho Corporation
3-30-13 Hongo, Bunkyo-ku
Tokyo 113-0033
JAPAN

