



5085 Corporate Exchange Blvd. S.E.  
Grand Rapids, MI 49512 • 616.656.2900

**PN 6-010-52-1**

## WRAP AROUND BUCKLE STRAP - INSTRUCTIONS FOR USE

Use the wrap around buckle strap to secure the patient on the surgical table during surgery. These straps are used with surgical tables that have side rails.



### **WARNING**

**Follow the recommended practices for the use of restraints to prevent patient injuries during surgery.**

Length	65 inches
	1651 millimeters
Width	2.5 inches
	63.5 millimeters
Weight	0.9 pounds
	0.41 kilograms



### **1. Wrap Around Buckle Strap Installed**

## Installation

To install the wrap around buckle strap, follow the directions below:

1. Feed one end of the strap under the rail on the surgical table.



### 2. Strap Under the Rail

2. Bring the strap up to wrap around the rail.
3. Feed the end of the strap through the bottom of the buckle.



### 3. Strap Through the Buckle

4. Feed the end of the strap through the top of the buckle.



### 4. Strap Through the Top of the Buckle

5. Pull the strap tight.



### 5. Strap Installed on a Rail

6. Repeat steps 1-5 to install the other end of the strap onto the rail on the other side of the surgical table.

Once the patient is on the surgical table and in position, complete the next steps.

7. Lift the fastener in the middle of the straps.



#### 6. Lift the Buckle Fastener

8. Slide the strap through the fastener.
9. Pull the straps tight.
10. Release the fastener.
11. Use the velcro to adhere the loose end of the strap to the taunt strap.

### Preventive Maintenance

Complete the preventive maintenance identified below. Regular preventive maintenance makes sure the accessory operates correctly. Preventive maintenance should only be completed by qualified and trained personnel from Skytron.

- Complete a general, visual examination during cleaning. Check for:
  - Loose parts
  - Frays
  - Tears

Replace the accessory if it is broken or shows excessive wear.

### Cleaning Recommendation



*Always follow the current Association of periOperative Registered Nurses (AORN) Journal Guidelines to make sure the proper cleaning and disinfection procedure is followed.*

Follow the procedures below to clean the strap between cases.



#### WARNING

**Always follow the standards for blood-borne pathogens from the Occupational Safety and Health Administration (OSHA). Use the recommended protective clothing, gloves, masks and eye protection to clean the table extension.**

1. Remove major contaminants from the accessory with disposable materials. Follow the appropriate biohazard waste disposal procedures.
2. Remove the accessory and place it on a flat surface for cleaning.

#### CAUTION

**Thoroughly read and follow the manufacturer's directions for all cleaning fluids. DO NOT use cleaners containing phenolics.**

3. Use a ready-to-use detergent diluted as required by the manufacturer. Make sure the active ingredients of the product are compatible with the materials used in the strap.
4. Apply cleaning fluid liberally to the strap.
5. Wipe with a clean, lint-free cloth.
6. Wipe until all the moisture and cleaning fluid is removed from the accessory.
7. Allow the accessory to dry.

## Disinfection

The following antiseptic solutions are approved for use on the straps:

- Sodium Hypochlorite 6% diluted to 0.1% (halogen containing compound)
- HypoAlcohol (iodine color removing agent)
- Chlorhexidine (chlorhexidine gluconate 0.5%)
- Benzalkonium Chloride (invert soap 10%)
- Povidone Iodine
- Ethanol 80%
- Oxydol (hydrogen peroxide)
- Isopropyl Alcohol (IPA) 99.5%

Use the following steps to disinfect the accessory.

Apply disinfectant to a clean, lint-free cloth.

Wipe the accessory with the cloth.



*Make sure there is enough disinfectant on the cloth so the cloth does not dry out before the entire accessory is wiped and disinfected.*

## Labels



**REVISION HISTORY**

<b>Date</b>	<b>Revision</b>	<b>Revision History</b>
3/17/2016	0	Initial Release

Distributed by:

SKYTRON

5085 Corporate Exchange Blvd. S.E.

Grand Rapids, MI 49512

616-656-2900

[www.skytron.us](http://www.skytron.us)

Manufactured by:



Professional Specialties

2224 E Person Ave

Memphis, TN 38114

901-452-7485



5085 Corporate Exchange Blvd. S.E.  
Grand Rapids, MI 49512 • 616.656.2900