P.N. B1-530-30

AURORA SERIES PRECISION HD STERILIZABLE COVER

INSTRUCTIONS FOR USE:

Aurora Series Precision HD Sterilizable Covers slide onto Aurora lightheads using a camera or counterweight providing sterile control for maneuverability of SKYTRON surgical lights.

Install cover over Precision HD Camera or counterweight. (See Figure 1)



Figure 1.

Ensure both engagement buttons have properly engaged the camera cover. Pull down on the camera cover after both buttons have engaged into place to ensure proper engagement. An audible "click" can be heard. (See Figure 2)



Figure 2.



Only use with Skytron Aurora lightheads using a Precision HD Camera or counterweight.

Always be sure the sterile handle is properly secured before using the lighthead. An improperly installed handle could fall out, resulting in possible injury to the patient or surgical staff.

Handles are subject to normal wear and tear. Always examine the handles for wear or damage to ensure proper and safe operation with the surgical light. Avoid use if any deformities or imperfections are found.

RECOMMENDED STERILIZATION PARAMETERS:

The Skytron sterilizable camera covers for the Skytron surgical lights are made of heat resistant, impact resistant plastic that will stand up to the rigors of the automatic washer. The cleaner used should be of a mild alkaline type without active chlorine and must be thoroughly rinsed after prescribed cleaning time.

Steam Sterilization

132°C (270°F): for 4 minutes

Always consult the AAMI and sterilizer manufacturer recommendations for proper sterilization procedures.

Replacement Parts:	None
Typical Dimensions:	Width: 3 5/8" [92mm] Length: 5 1/8" [130mm]
Weight:	.40lb [.18kg]

Date	Revision	Revision History
07/11/2018	0	Initial release.
06/20/2019	1	Removed CE from bottom of document.

Authorized Distributors:

U.S. - SKYTRON

5085 Corporate Exchange Blvd. S.E. Grand Rapids, MI 49512

EUROPE - SKYTRON EUROPE B.V.

FLORESSTRAAT 52 8022 AD ZWOLLE THE NETHERLANDS

EC REP

Manufactured by:

DKK Dai-Ichi Shomei Co., LTD 32-26 Sakashita 1-Chome, Itabashi-Ki, Tokyo 174-0043 JAPAN



