


DOVERA LIGHT HANDLE ADAPTER - INSTRUCTIONS FOR USE

The DoVera light handle adapter attaches to the DoVera light, and is designed to fit single use handle covers.

The Kit Includes

- Replacement handle
- Hex wrench
- Set screws

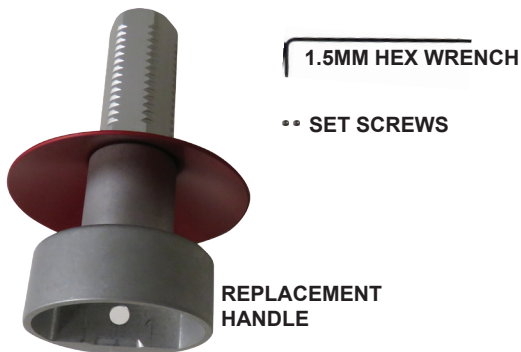

Assembly Instructions

1. Rotate the lighthead so the handle is facing upward to facilitate installation.
2. Remove the handle cover (if present). Press the lock button and pull it off.

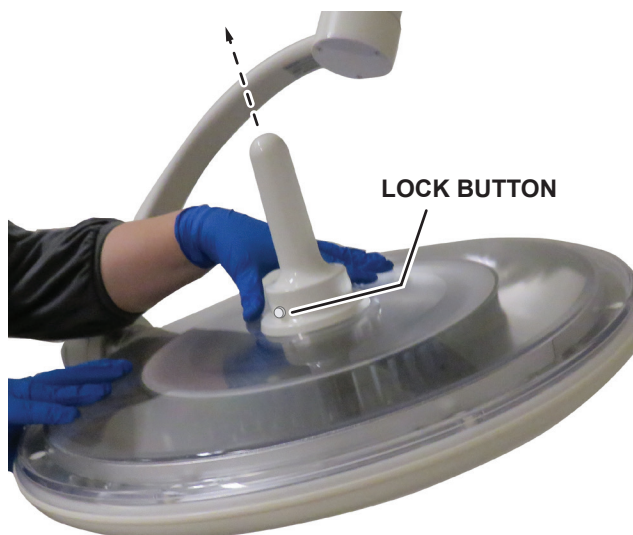


Figure 1. Remove the handle cover

3. Attach the handle adapter. Slide it onto the post until the lock button locks it into place.



Figure 2. Install the handle adapter on the post

4. Secure the handle adapter in place. Tighten the two set screws with the included 1.5mm hex wrench.

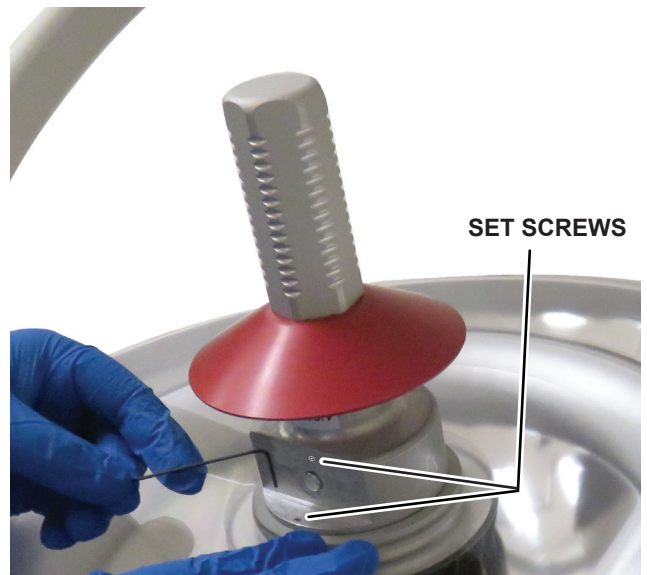


Figure 3. Tighten the set screws

5. Rotate the lighthead so the newly installed handle is facing down.
6. Inspect to ensure the newly installed handle adapter is securely installed.


Revision History

Date	Revision	Revision History
02/11/2020	0	Initial Release

Distributed by:

US - SKYTRON
5085 Corporate Exchange Blvd. S.E.
Grand Rapids, MI 49512 (616) 656-2900
www.skytron.us

Manufactured by:

 Litex Manufacturing
820 Thompson Ave #9
Glendale, CA 91201
United States

The base language for this document is ENGLISH. Any translations must be from the base language document.

Printed copies are not controlled documents.

Although current at the time of publication, SKYTRON'S policy of continuous development makes this document subject to change without notice. If a current document is required, contact your local SKYTRON representative or contact SKYTRON directly at the distribution addresses listed above.

