




5085 Corporate Exchange Blvd. S.E.
Grand Rapids, MI 49512 • 616.656.2900

PN H8-300-60-A

REAR SHELF MOUNTING BRACKET - INSTRUCTIONS FOR USE

The rear shelf mounting bracket attaches to the rear channel on SKYTRON Ergon 2 or 3 carriers and is used for mounting a shelf or other accessories.

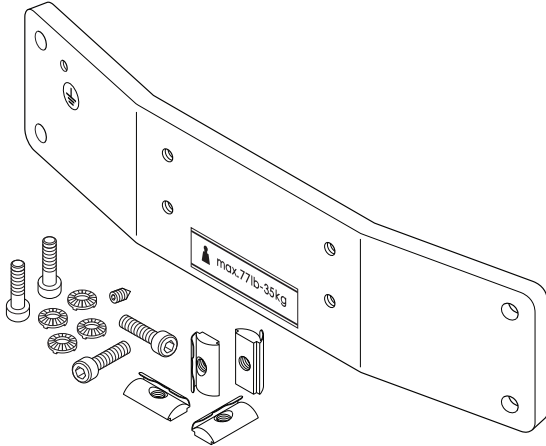

Typical Dimensions

- Length 9-7/8" [251.27mm]
- Width 2-3/8" [60mm]
- Thickness 5/16" [8mm]
- Max Load 77lbs [35kg]

Installation

1. Determine the approximate location on the carrier you wish to attach the mounting bracket.
2. Press two T-nuts into each slot on the back of the carrier and rotate it into position (Figure 1).

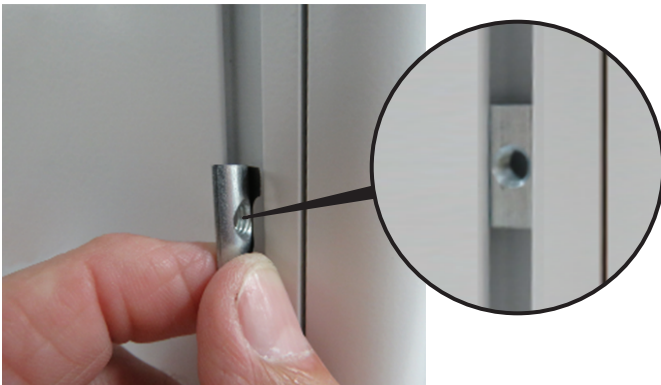


Figure 1. Insert the T-Nut

3. Align the four holes in the mounting bracket with the T-nuts and secure to the carrier using the supplied Allen head bolts and lock washers. (Figure 2).


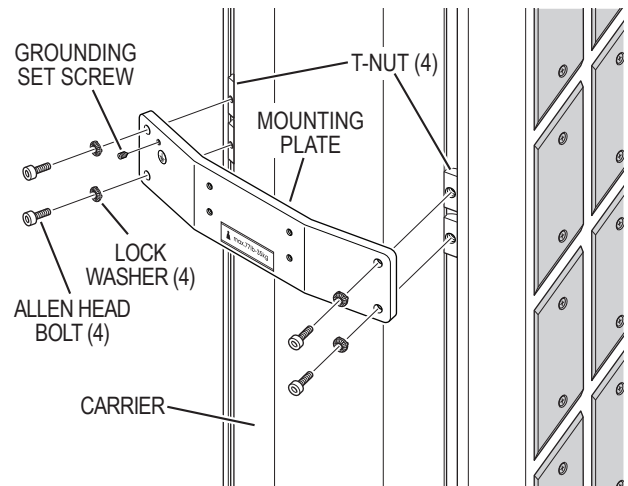


Figure 2. Attach The Mounting Bracket

4. Install the grounding set screw in the threaded hole indicated by the grounding label.

Labels

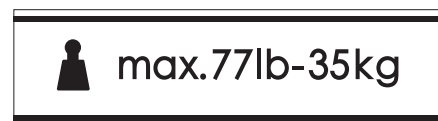

Date	Revision	Revision History
09/16/2016	0	Initial Release

Distributed by:

US - SKYTRON
5085 Corporate Exchange Blvd. S.E.
Grand Rapids, MI 49512 (616) 656-2900
www.skytron.us

Manufactured by:



IHBO BV
Floresstraat
NL-8022 AD Zwolle

The base language for this document is ENGLISH. Any translations must be from the base language document.
Printed copies are not controlled documents.

Although current at the time of publication, SKYTRON'S policy of continuous development makes this document subject to change without notice. If a current document is required, contact your local SKYTRON representative or contact SKYTRON directly at the distribution addresses listed above.



5085 Corporate Exchange Blvd. S.E.
Grand Rapids, MI 49512 • 616.656.2900 • FAX 616.656.2906