

## H9-200-05-2 TETHER BRACKET INSTALLATION - INSTRUCTIONS FOR USE

This document describes how to install the H9-200-05-2 tether bracket to an Ergon 3 carrier, for use with anesthesia machines containing GCX style channels.


### Required Tools

- Impact screwdriver
- #3 Phillip's head bit
- 5mm Allen wrench

### Install the Bracket Adapter to the Carrier


Prior to assembling the bracket, attach the bracket adapter to the carrier.

1. Remove the plate on the bottom of the carrier (Figure 1). Use an impact screwdriver with a #3 Phillip's head bit to remove the four corner screws holding the plate in place. Hold the plate up so it does not drop on the floor.

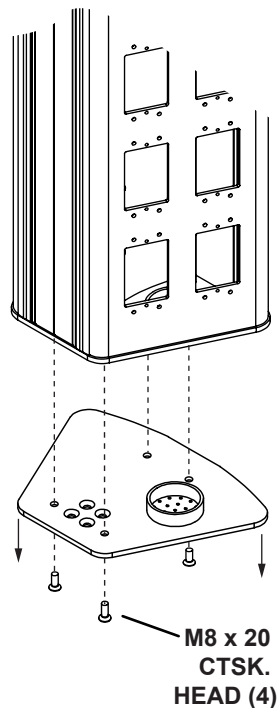


**Figure 1. Remove the bottom plate**

2. Attach the bracket adapter to the bottom plate (Figure 2). Align the four holes at the top of the adapter with the four holes on the bottom plate. Use a 5mm Allen wrench to secure the adapter to the plate with four M8 x 50 countersunk screws.

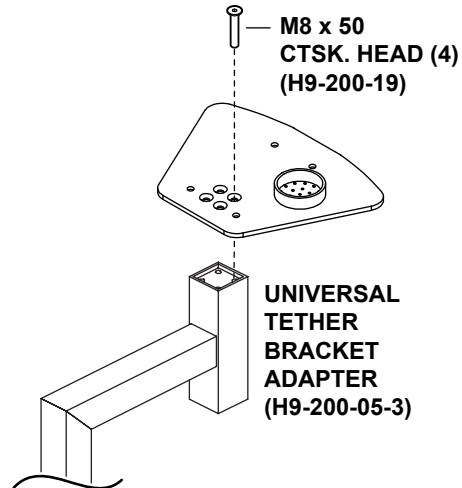

**Figure 2. Universal tether bracket adapter**

3. Attach the plate to the bottom of the carrier (Figure 3). Align the four corner holes with the holes at the bottom of the carrier. Use an impact driver with a #3 Phillip's head bit to screw the plate to the carrier with four M8 x20 countersunk screws.

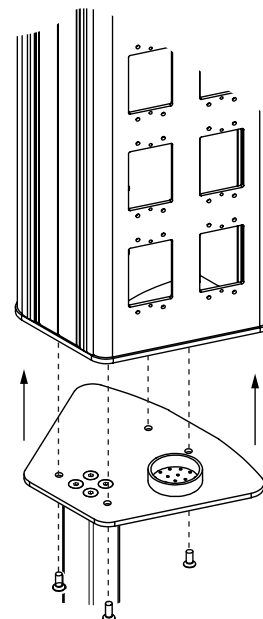

**Figure 3. Attach the bottom plate**

## Assemble the Tether Bracket

1. Slide the Universal Tether Tee into the Universal Tether Bracket Adapter (Figure 4).
2. Insert the Universal Tether Mounting Bracket into base of the Universal Tether Tee (Figure 4). Adjust it to the desired position.
3. Screw a knob into the threaded hole in the Universal Tether Tee (Figure 4) and tighten it to secure the Universal Tether Mounting Bracket.
4. Slide the GCX adapter plate into the rail on the anesthesia machine.
5. Align the Mounting Bracket so the hole in the end of the Mounting Bracket arm aligns with the hole in the GCX Adapter Plate.
6. Screw a knob into the bracket arm and GCX Adapter Plate (Figure 4).

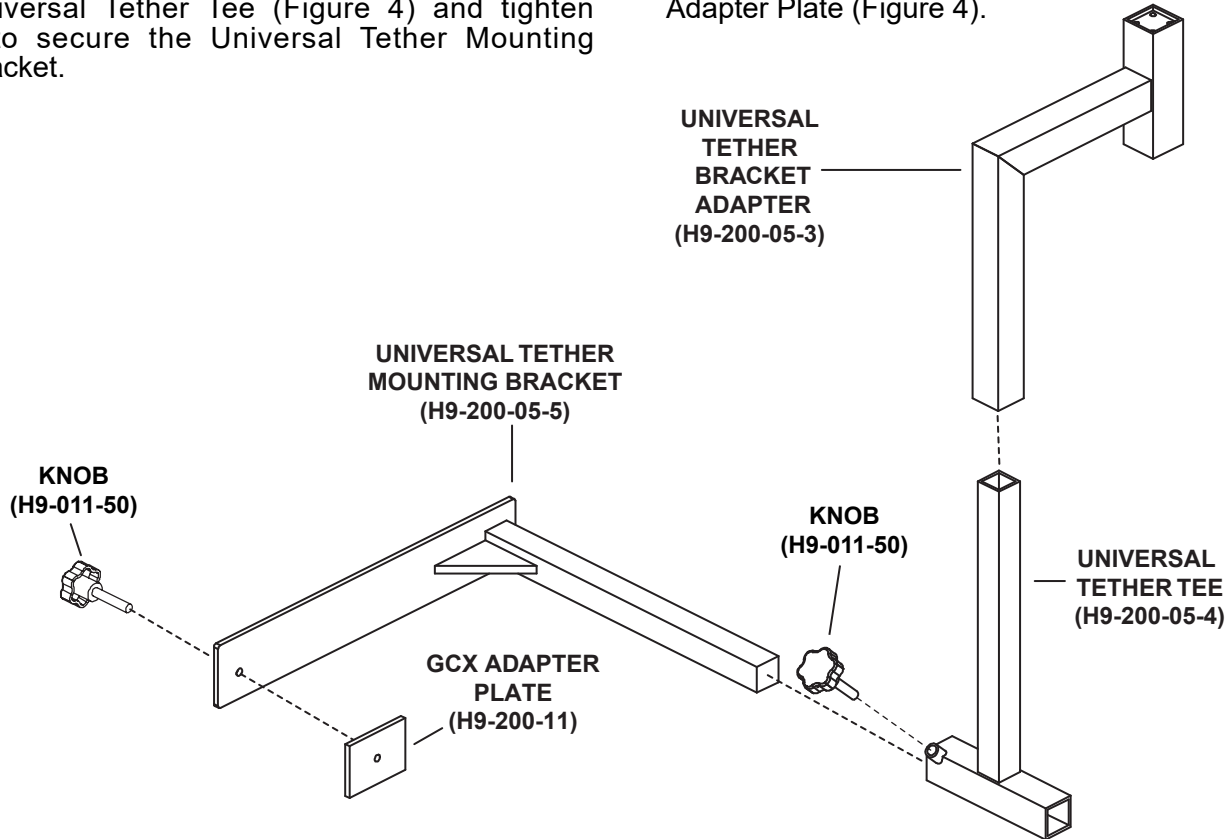


Figure 4. Tether bracket assembly

## Revision History

Date	Revision	Revision History
01/10/2018	0	Initial Release

### Distributed by:

US - SKYTRON  
5085 Corporate Exchange Blvd. S.E.  
Grand Rapids, MI 49512 (616) 656-2900  
www.skytron.us

### Manufactured by:

Accuform Industries Inc.  
1701 Broadway, N.W.  
Grand Rapids, MI 49504

