



5085 Corporate Exchange Blvd. S.E.
Grand Rapids, MI 49512 • 616.656.2900

PN H8-300-51-A

CUFF HOLDER MOUNT - INSTRUCTIONS FOR USE

The cuff holder mount is made of stainless steel and specifically designed for the Welch Allyn 767 mobile aneroid blood pressure cuff (Not supplied by SKYTRON). It attaches to any equipment rail and features a quick release push button for fast and easy attachment.



Typical Dimensions

Height: 4-1/4" [108mm]

Width: 5-1/2" [140mm]

Installation

1. Align the three bolt holes on cuff holder with the rail mount.
2. Insert the first screw through the top corner of cuff holder and rail mount with the screw head facing inside cuff holder (Figure 1).

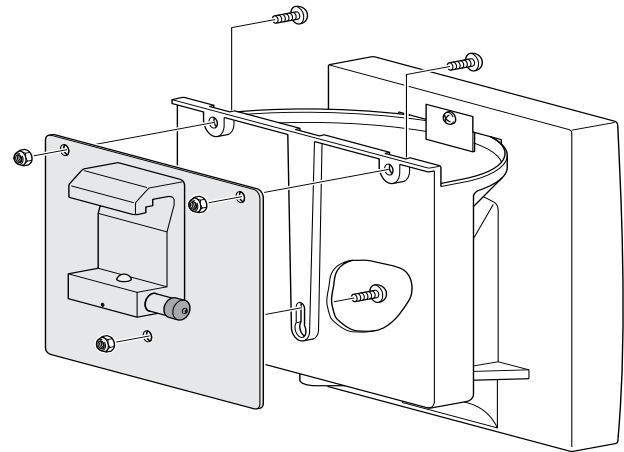


Figure 1. Cuff holder and Rail Mount

3. Thread one lock nut on to the screw and finger tighten.
4. Install the remaining two screws and lock nuts.
5. Using 90 degree Phillips head screwdriver and 8mm wrench tighten screws.

CAUTION

Avoid overtightening the screws as damage could occur to cuff holder.

Attaching The Cuff To The Equipment Rail



WARNING

Edges of the rail mount could tear medical gloves exposing bare skin. Use caution when installing, removing or repositioning the blood pressure cuff on the equipment rail.

1. Place the rail mount clamp over the equipment rail at approximately a 45 degree angle.
2. Press the lock button and lower the cuff holder on to the rail (Figure 2).
3. Release the Lock Button and ensure the cuff holder is properly secured to the equipment rail.

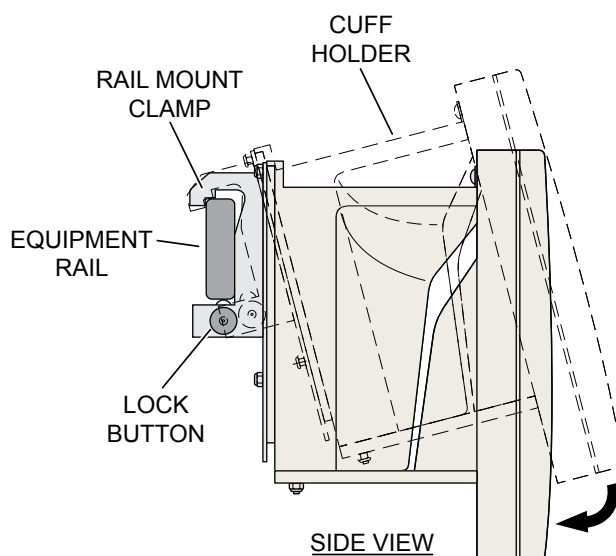


Figure 2. Attaching the Cuff Holder

Date	Revision	Revision History
04/11/2016	0	Initial Release

Distributed by:

US - SKYTRON
5085 Corporate Exchange Blvd. S.E.
Grand Rapids, MI 49512 (616) 656-2900
www.skytron.us

Manufactured by:

 Welch Allyn, Inc
4341 State Street Road
Skaneateles Falls, NY 13153
www.welchallyn.com

NOTICE

This instruction for use represents a typical Welch Allyn cuff holder. Variations and changes may exist. Please consult with your Welch Allyn representative for further details.

The base language for this document is ENGLISH. Any translations must be from the base language document.

Printed copies are not controlled documents.

Although current at the time of publication, SKYTRON'S policy of continuous development makes this manual subject to change without notice. If current manuals are required, contact your local SKYTRON representative or contact SKYTRON directly at the distribution addresses listed above.

 **SKYTRON®**
5085 Corporate Exchange Blvd. S.E.
Grand Rapids, MI 49512 • 616.656.2900 • FAX 616.656.2906