

Service Bulletin

SB # 246

Published: 11/06/2020

By: KHO/DJ


- ☐ Generally
- ☐ Dishwashers
- ☐ Bedpan Washer Disinfectors
- ☒ Instrument Washer Disinfectors
- ☐ Process Solution

- ☐ To be exchanged as soon as possible
- ☐ To be exchanged at the next maintenance
- ☐ To be exchanged at the usual service interval
- ☒ Information


Change made from machine # N/A

IQ5 & IQ6 - Troubleshooting door gasket

1. Make sure that the machine is leveled correct, and control the diagonal measurement.

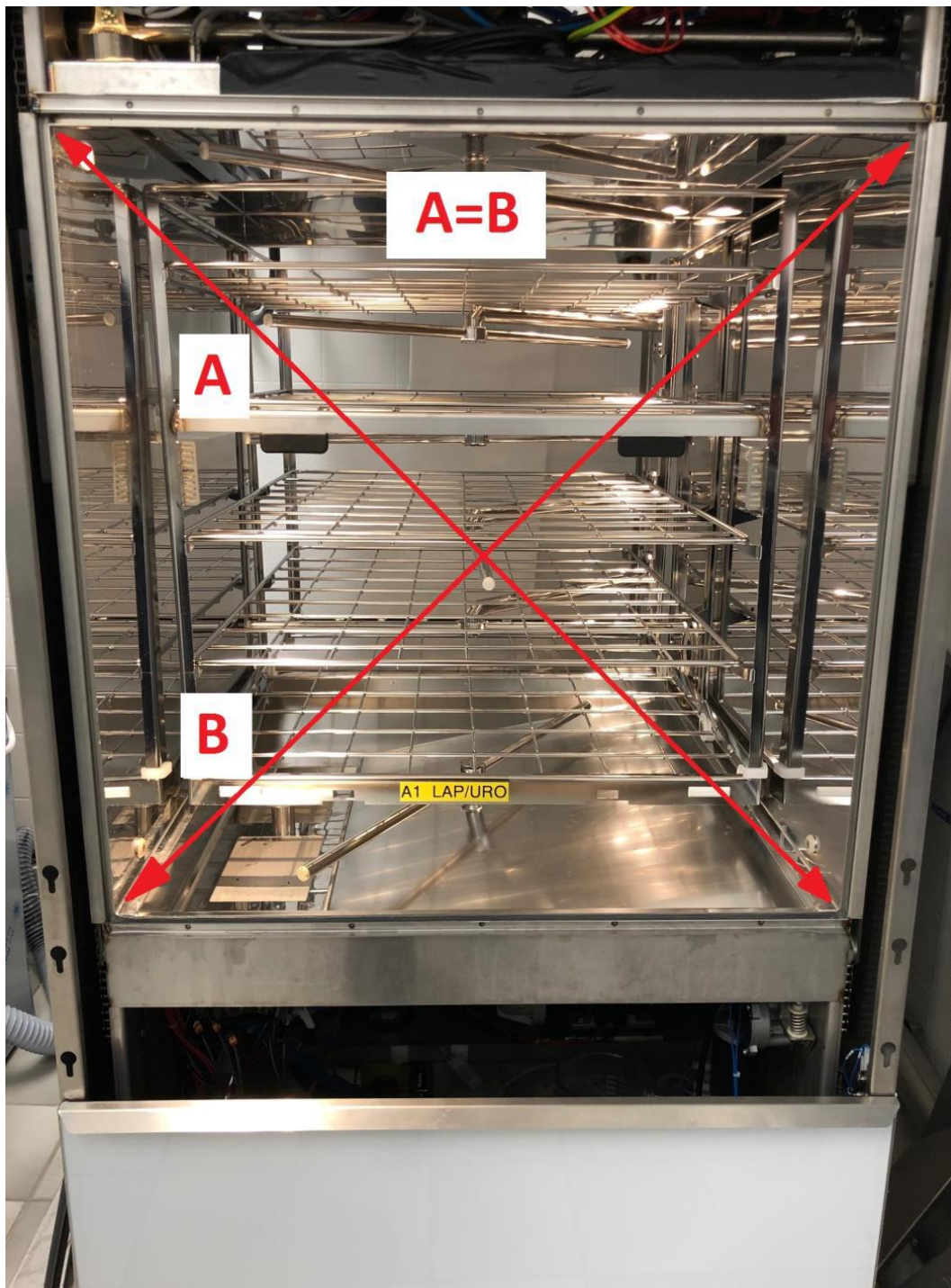

2. Check that the groves for the door gasket are clean and applied wit silicone sealant.

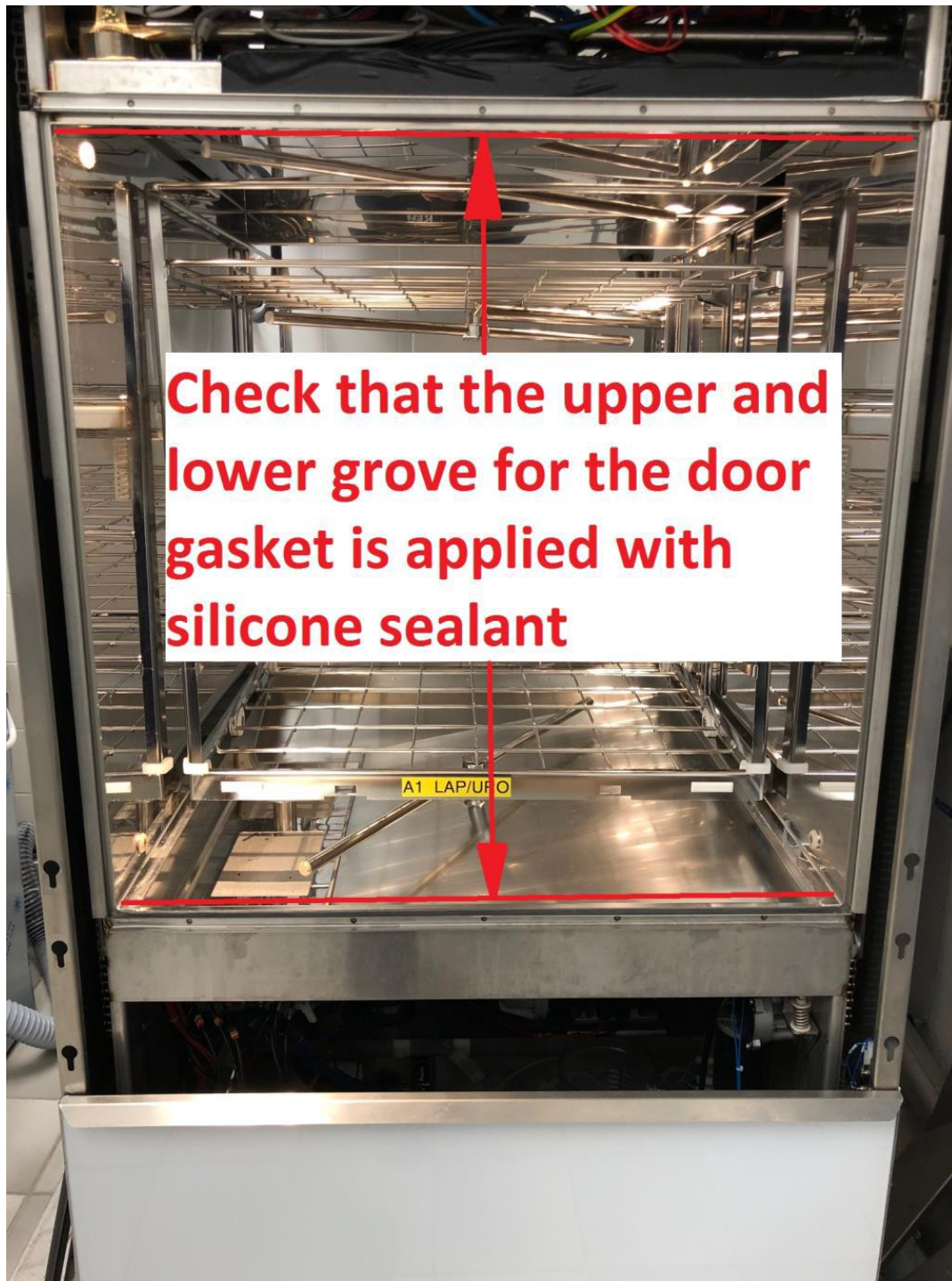


Service Bulletin

SB # **246**
Published: **11/06/2020**

By: KHO/DJ

If the sealant is damaged or missing, then clean the groove and apply a thin layer of silicone sealant

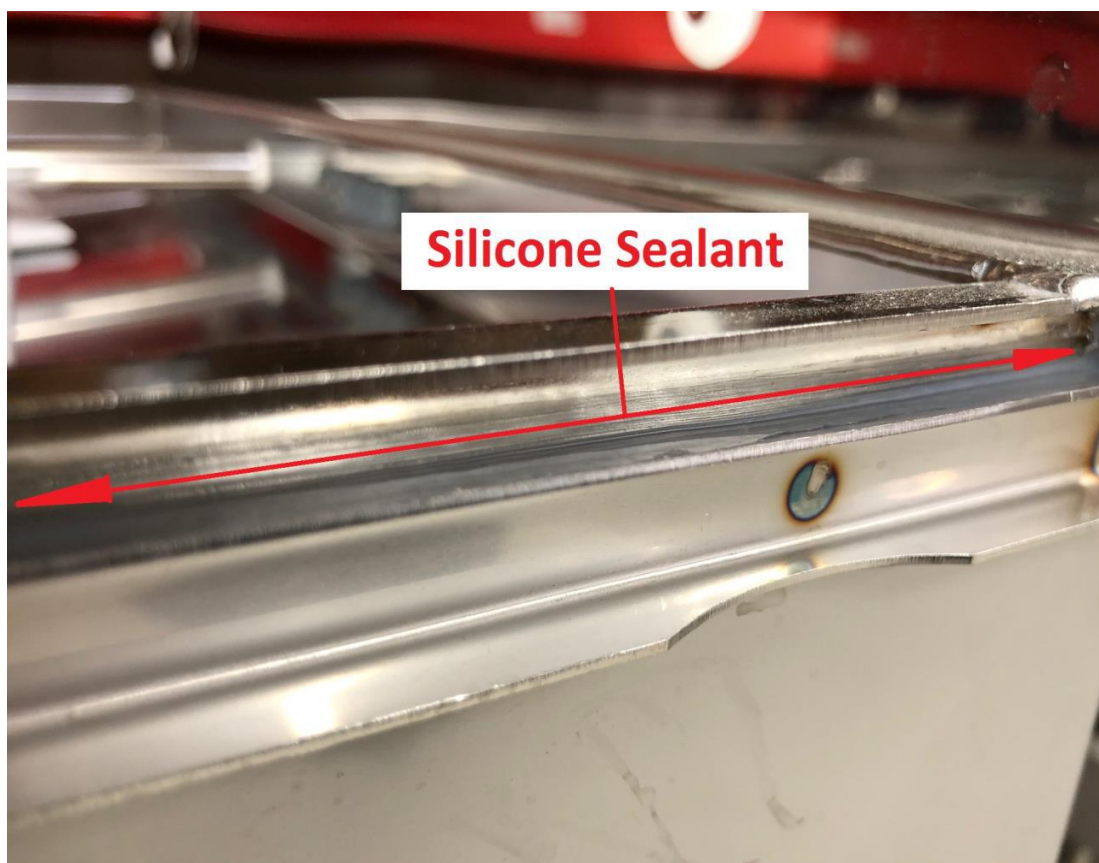


Service Bulletin

SB # 246

Published: 11/06/2020

By: KHO/DJ

3. Check if the chamber is bending in or outwards. If it is not within the limits shown below, then it may be carefully adjusted with a nylon or rubber hammer.

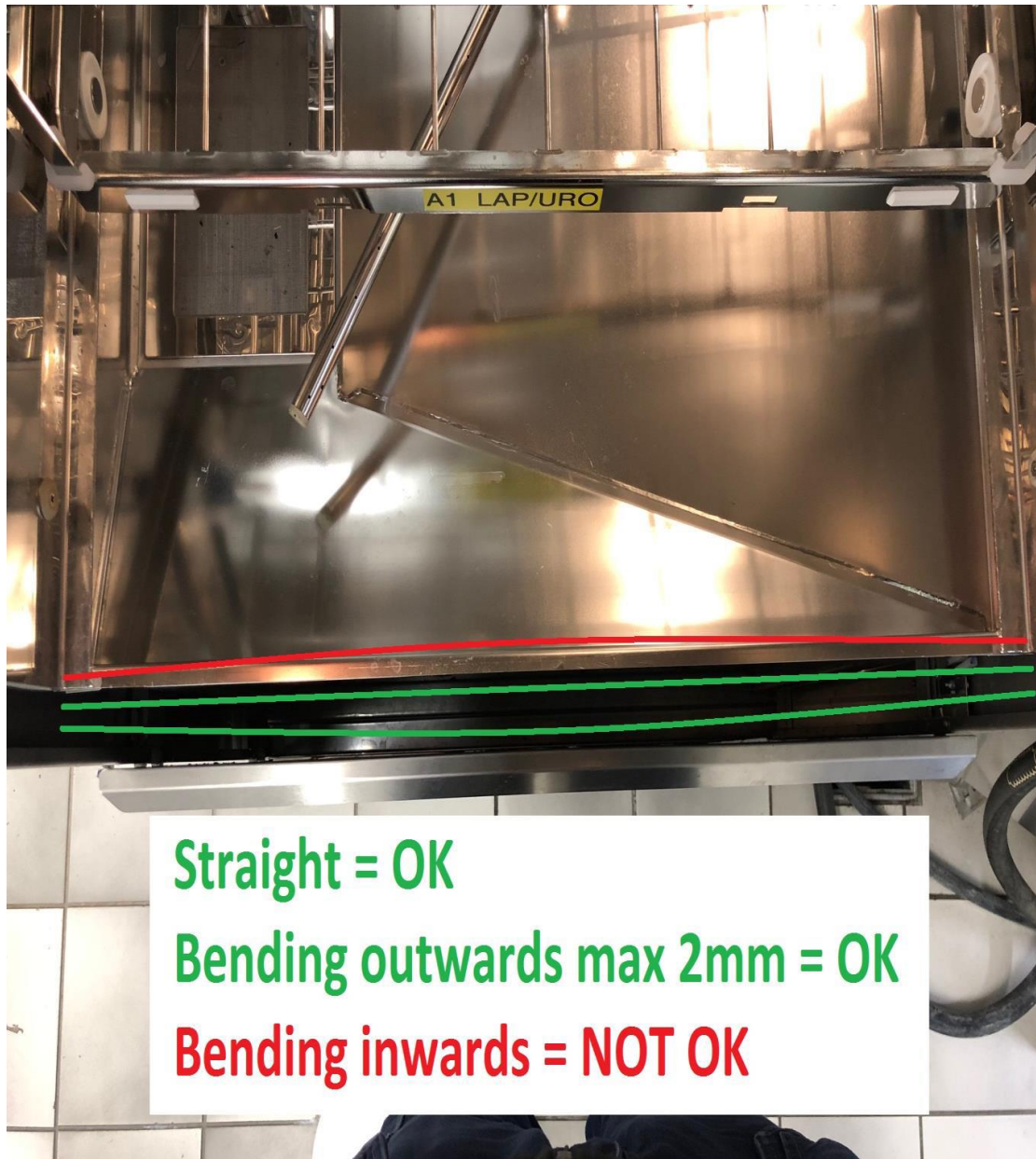

Service Bulletin

SB # 246

Published: 11/06/2020

By: KHO/DJ

4. Check the width of the groove for the door gasket. The groove must be between 7,5 and 8,5 mm in width. If the groove is larger than 8,5 mm in width, there must be applied part # 200121, which is a piece of sheet metal with adhesive tape.



Service Bulletin

SB # 246

Published: 11/06/2020

By: KHO/DJ



These instructions are intended as guidance only, to help certified technicians, trained by KEN HYGIENE SYSTEMS A/S, or by a KEN HYGIENE SYSTEMS A/S approved training partner, install and maintain products manufactured by KEN HYGIENE SYSTEMS A/S.
Rights to alterations reserved!

KEN HYGIENE SYSTEMS A/S, DK- 5672 BROBY, CVR 10676096 Ph.: +4562631091 E-mail: Service@ken.dk Internet: www.kn.dk