

Service Bulletin

SB # 216

Published: 25/11/2019

By: Service/DJ



- ☐ Generally
- ☐ Dishwashers
- ☐ Bedpan Washer Disinfectors
- ☒ Instrument Washer Disinfectors
- ☐ Process Solution

- ☐ To be exchanged as soon as possible
- ☐ To be exchanged at the next maintenance
- ☐ To be exchanged at the usual service interval
- ☒ Information



Change made from machine #

IQ5 & IQ6 Installation instructions condenser basic


Place the tank and cover on top of the machine, secure it with bolts and nuts.

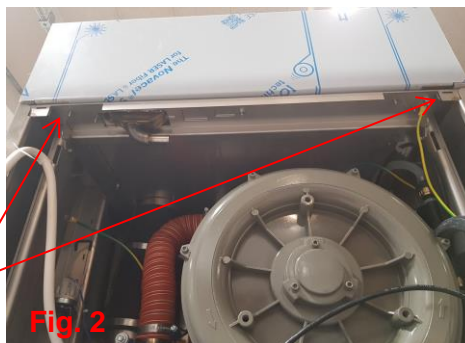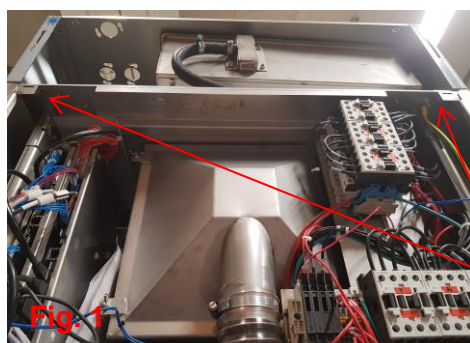

Locate the bundle with solenoid valves in the upper service area, pull them up and forward, mount them as shown on fig 5 and 6 below. Valve V1 wire 20-6 / 3L2 to the left, valve V2 wire 20-7 / 3L2 to the right and valve V4 wire 20-13 / 3L2 in the front.

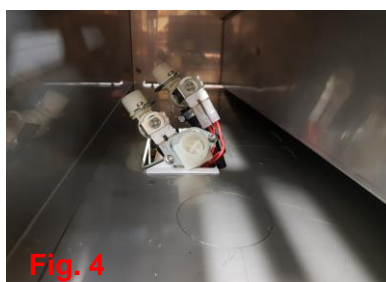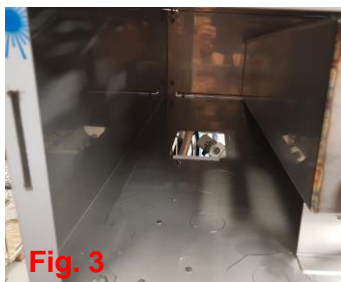

Mount the exhaust tube as shown on fig. 7, and the drain tube as shown on fig. 8 below.

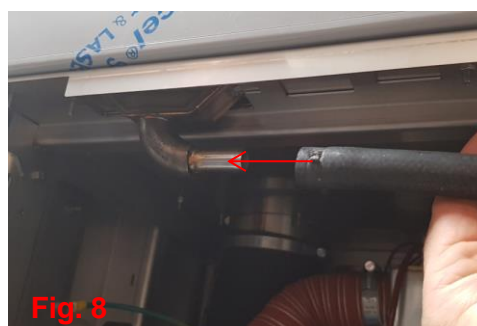

Locate the mains supply cable, mount the lead in bushing in the top plate and lead the cable through.



These instructions are intended for guidance purposes only. They have been developed to support authorised technicians, trained by KEN HYGIENE SYSTEMS A/S, or by a KEN HYGIENE SYSTEMS A/S approved training partner, with the installation and maintenance of, products manufactured by KEN HYGIENE SYSTEMS A/S.

The instructions are subject to change without notice.