

Service Bulletin

SB # **248**

Published: **21/09/2020**

By: Service/DJ

- ☐ Generally
- ☐ Dishwashers
- ☐ Bedpan Washer Disinfectors
- ☒ Instrument Washer Disinfectors
- ☐ Process Solution

- ☐ To be exchanged as soon as possible
- ☐ To be exchanged at the next maintenance
- ☐ To be exchanged at the usual service interval
- ☒ Information

Change made from machine # N/A

Setup for touch panels on IQ5 or IQ6:


Adjust the DIP-switches for loading or unloading side, refer to fig. 1 or 2 below.

- Insert a micro SD card or USB memory stick with the correct "KUPGRADE" file.
- Power ON the machine, the display will automatically start up and show "Upgrading".
- Wait until the display shows "Done".
- Power OFF the machine.
- Remove the micro SD card or USB memory stick.
- Power ON.




HMI loading side

Fig. 1

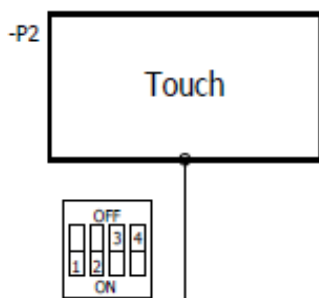

HMI unloading side

Fig. 2

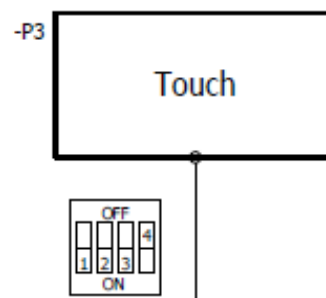

These instructions are intended for guidance purposes only. They have been developed to support authorised technicians, trained by KEN HYGIENE SYSTEMS A/S, or by a KEN HYGIENE SYSTEMS A/S approved training partner, with the installation and maintenance of, products manufactured by KEN HYGIENE SYSTEMS A/S.

The instructions are subject to change without notice.